RadiForce® MX191

Color LCD Monitor

English

Deutsch

Français

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Instructions for Use: ■ Setup Manual

Important: Please read PRECAUTIONS, this Setup Manual and the User's Manual (on the CD-ROM) carefully to familiarize yourself with safe and effective usage. Please retain this manual for future reference.

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The monitor supports the following resolutions.

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* Recommended resolution (Set this resolution)

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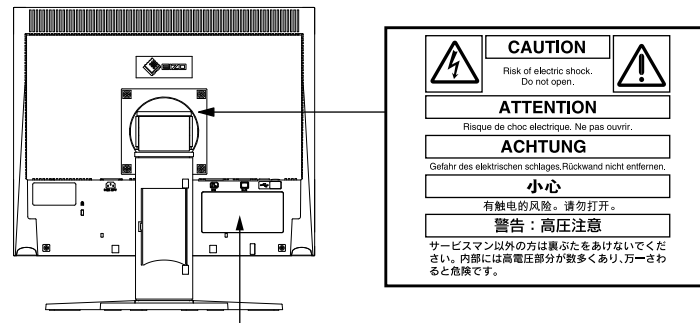
RadiForce® MX191

Color LCD Monitor

[Location of the Caution Statements]

The equipment must be connected to a grounded main outlet.
Jordet stikkontakt skal benyttes når apparatet tilkobles datanett.
Apparaten skall anslutas till jordat nätuttag.
電源コードのアースは必ず接地してください。
这设备必须连接至接地主插座。

Symbols on the unit

Symbol	This symbol indicates
	Power Button Press to turn the monitor's power on or off.
	Alternating current
	Alerting electrical hazard
	Caution Refer to SAFETY SYMBOLS section in PRECAUTIONS.

No part of this manual may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, or otherwise, without the prior written permission of EIZO Corporation. EIZO Corporation is under no obligation to hold any submitted material or information confidential unless prior arrangements are made pursuant to EIZO Corporation's receipt of said information. Although every effort has been made to ensure that this manual provides up-to-date information, please note that EIZO monitor specifications are subject to change without notice.

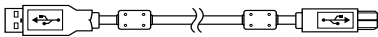
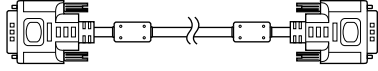
Package Contents

Check that all the following items are included in the packaging box. If any items are missing or damaged, contact your local dealer.


NOTE

Please keep the packaging box and materials for future movement or transport of the monitor.

- Monitor
- Power cord
- Digital signal cable (DD300)
- EIZO LCD Utility Disk (CD-ROM)
 - RadiCS LE (for Windows)
 - ScreenManager Pro for Medical (for Windows)
 - User's Manual
- Setup Manual (this manual)
- PRECAUTIONS
- Recycling Information
- Mounting Screws (M4 x 12mm, 4 pcs)

Controls and Functions

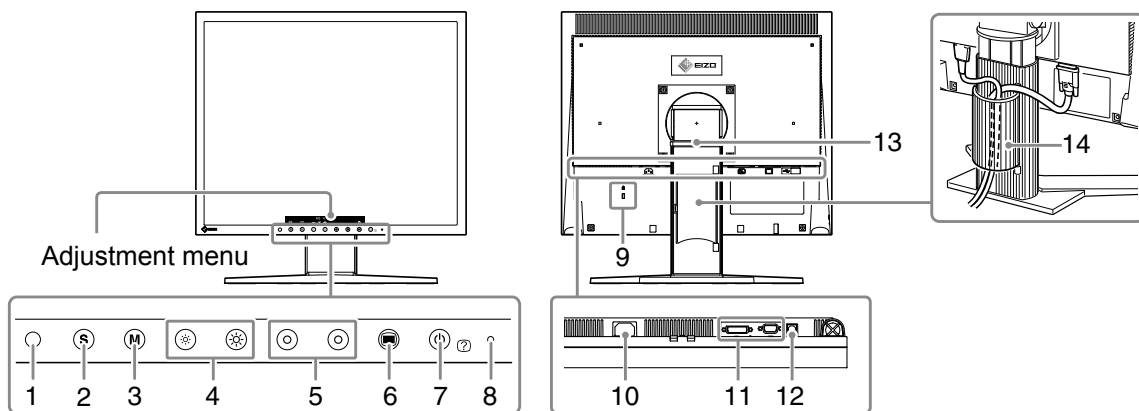1	Sensor	Detects ambient brightness. Auto EcoView function.
2	Ⓢ button	Switches input signals for display when two PCs are connected to the monitor.
3	Ⓜ button	Allows you to switch the display mode. Pressing the mode button repeatedly switches the setting of the CAL switch mode.
4	☀️☀️ buttons	Adjusts the brightness.
5	⦿ button	Chooses an adjustment item or increases/decreases adjusted values for advanced adjustments using the Adjustment menu.
6	Ⓜ button	Displays the main menu. By pressing the menu button while the menu is displayed, you can return to the previous status of the menu.
7	Ⓢ button	Turns the power on or off. By pressing the Power/Help button while the menu is displayed, the function names of each icon are displayed.
8	Power indicator	Indicates monitor's operation status. Blue: Operating Orange: Power saving Off: Power off
9	Security lock slot	Complies with Kensington's MicroSaver security system.
10	Power connector	Connects the power cord.
11	Input signal connectors	Left: DVI-D connector / Right: D-sub mini 15-pin connector.
12	USB port (Up)	Connects the USB cable to use the software that needs USB connection.
13	Stand	Used to adjust the height and angle of the monitor screen.
14	Cable holder	Covers the monitor cables.

Connecting Cables

Note

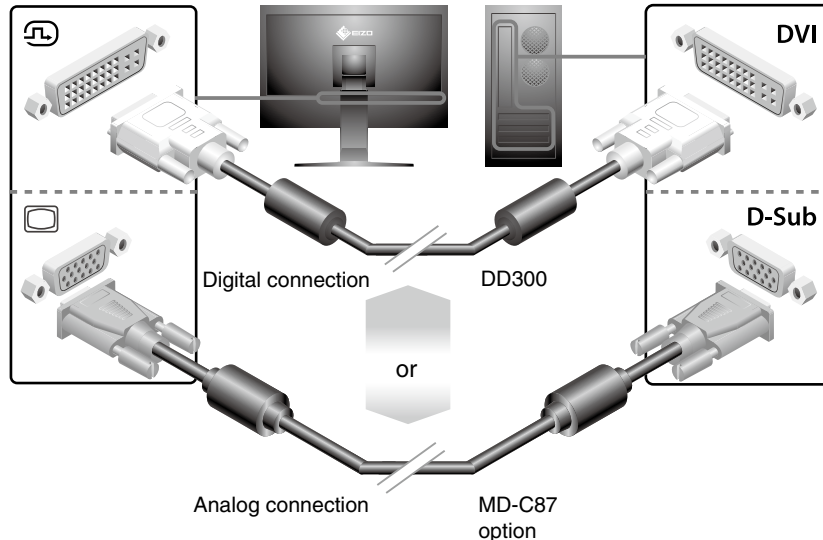
- Check that the monitor and the PC are powered off.
- When replacing the current monitor with a MX191 monitor, be sure to change the PC settings for resolution and vertical frequency to those which are available for the MX191 monitor referring to the resolution table (back of cover page) before connecting the PC.

Tips

- When connecting two PCs to the monitor, refer to the User's Manual on the CD-ROM.

1 Connect the monitor to the PC with a signal cable that matches the connectors.


After connecting the cable connectors, tighten the screws of the connectors to secure the coupling.

2 Connect the USB cable when using RadiCS LE or ScreenManager Pro for Medical.

Connecting to the Monitor

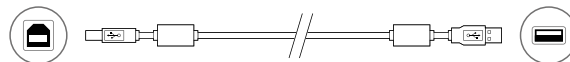Connecting to a Computer

3 Plug the power cord into a power outlet and the power connector on the monitor.

4 Press to turn on the monitor.

The monitor's power indicator lights up blue.

5 Turn on the PC.

The screen image appears.

When turning on the monitor and PC for the first time with the analog signal, the Auto Adjustment function works to adjust the clock, phase, and display position automatically.

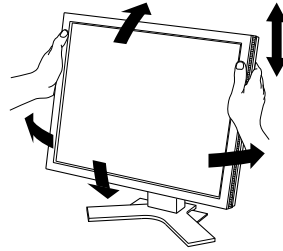
If an image does not appear, refer to "No-Picture Problem" (page.6) for additional advice.

Note

- Turn off the monitor and PC after using them.
- For the maximum power saving, it is recommended that the Power button be turned off. Unplugging the power cord completely shuts off power supply to the monitor.

Adjusting the Screen Height and Angle

Hold left and right edge of the monitor with both hands, and adjust the screen height, tilt and swivel of the screen to the best condition for working.

Note

- Be sure that the cables are correctly connected.
-

Installing RadiCS LE

The quality control software "RadiCS LE" (for Windows) is included with this monitor. To calibrate the monitor and manage the calibration history, install RadiCS LE with reference to the RadiCS LE User's Manual on the CD-ROM.

Note

- Connect the monitor to the PC with a USB cable when using RadiCS LE.
-

Tips

- To perform advanced quality control that conforms to medical standards, use the optional monitor quality control software kit "RadiCS UX1".
-

Selecting Display Mode

CAL Switch function allows you to select the best display mode easily according to monitor's application.

For details of CAL Switch function, refer to the User's Manual on the CD-ROM.

CAL Switch Mode

 (DICOM-CL)	DICOM-CL mode. Available for setting tailored to X-ray film color (clear base).
 (TXT)	Text mode. Suitable for displaying texts for word processing or spreadsheets.
 (CUS)	Custom mode. Available for the color settings according to your preference.
 (CAL)	CAL mode. Displays the screen adjusted by calibration software.

1 Press .

The CAL Switch menu appears.

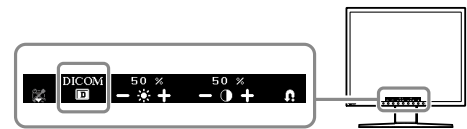
2 Press again while the adjustment menu is displayed.

Each time the button is pressed, the mode on the screen switches. (Refer to the CAL Switch Mode table.)

3 Select .

The selected mode is set.

Current CAL Switch mode

The basic settings and adjustments are completed. For advanced settings/adjustments, refer to the User's Manual on the CD-ROM.

No-Picture Problem

If no picture is displayed on the monitor even after the following remedial action is taken, contact your local dealer.

1. Check the Power indicator.

Symptom	Status	Possible cause and remedy
No picture	Power indicator does not light up.	<ul style="list-style-type: none"> • Check whether the power cord is connected correctly. If the problem persists, turn off the monitor, and then turn it on again a few minutes later. • Press .
	Power indicator lights blue.	<ul style="list-style-type: none"> • Set each adjusting value in [Brightness], [Contrast] and [Gain] to higher level.
	Power indicator lights orange.	<ul style="list-style-type: none"> • Switch the input signal with . • Press a key on the keyboard or click the mouse. • Check whether the PC is turned on.

2. Check the error message that remains on the screen.


These messages appear when the input signal is incorrect even if the monitor is functioning.

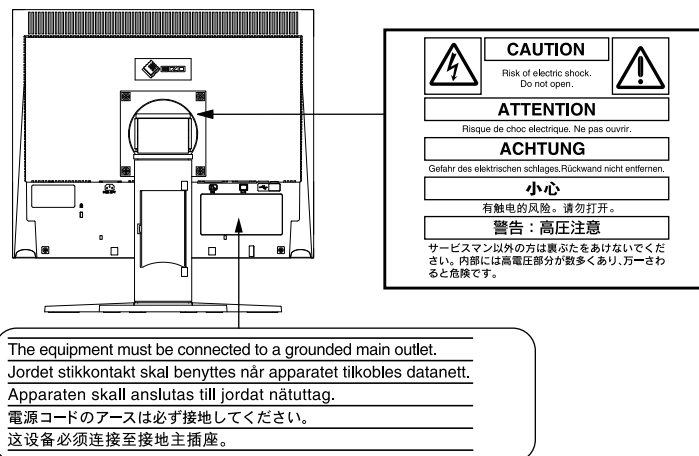Symptom	Status	Possible cause and remedy
	The input signal is not received.	<ul style="list-style-type: none"> • The message shown left may appear, because some PCs do not output the signal soon after power-on. • Check whether the PC is turned on. • Check whether the signal cable is properly connected to the PC. • Switch the input signal with .
	The message shows that the input signal is out of the specified frequency range. (Such signal frequency is displayed in yellow.)	<ul style="list-style-type: none"> • Check whether the signal setting of your PC matches to the resolution and the vertical frequency settings for the monitor. (See "Compatible Resolutions/Frequencies" on the cover back.) • Reboot the PC. • Change the mode to an appropriate mode using the graphics board's utility software. Refer to the manual of the graphics board for details. <p>fD: Dot Clock (Displayed only when the digital signal inputs) fH: Horizontal Frequency fV: Vertical Frequency</p>

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