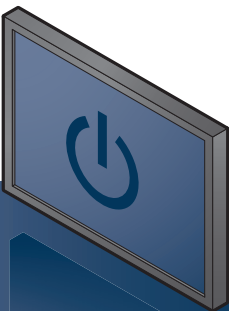
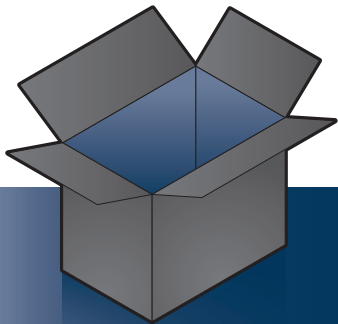
# RadiForce® MS235WT

Touch Panel Color LCD Monitor

Setup Guide Installationsanleitung Guide d'installation 设定指南

**Important :** Please read PRECAUTIONS, this Setup Guide and the User's Manual stored on the CD-ROM carefully to familiarize yourself with safe and effective usage. Please retain this guide for future reference.

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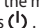
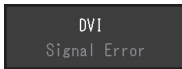
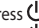
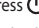
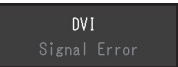
Menu

Color  
Screen  
Sound  
PowerManager  
Monitor Settings  
EcoView  
Information  
Exit

**No-Picture Problem** If no picture is displayed on the monitor even after the following remedial action is taken, contact your local EIZO representative.

Problem		Possible cause and remedy		Problem	Possible cause and remedy	
No picture	Power indicator does not light.	<ul style="list-style-type: none"><li>• Check whether the power cord is connected properly.</li><li>• Turn the main power switch on.</li><li>• Press .</li><li>• Turn off the main power, and then turn it on again a few minutes later.</li></ul>		No picture	Power indicator is flashing orange.	<ul style="list-style-type: none"><li>• The device that is connected using DisplayPort have a problem. Solve the problem, turn off the monitor, and then turn it on again. Refer to the User's Manual of the output device for further details.</li></ul>
	Power indicator is lighting blue.	<ul style="list-style-type: none"><li>• Increase "Brightness", "Contrast", or "Gain" in the Setting menu.</li></ul>		The message appears.	<div>The message shows that the input signal is out of the specified frequency range. Example: </div>	<ul style="list-style-type: none"><li>• Check whether the PC is configured to meet the resolution and vertical scan frequency requirements of the monitor.</li><li>• Reboot the PC.</li><li>• Select an appropriate setting using the graphics board's utility. Refer to the User's Manual of the graphics board for details.</li></ul>
	Power indicator is lighting orange.	<ul style="list-style-type: none"><li>• Switch the input signal with <b>SIGNAL</b>.</li><li>• Move the mouse or press any key on the keyboard.</li><li>• Check whether the PC is turned on.</li><li>• Depending on the PC and graphics board, the input signal is not detected and the monitor does not recover from the power saving mode. If the screen is not displayed even after moving the mouse or pressing any key on the keyboard, turn the monitor off and on using its power button. When the screen is displayed, execute the following procedure. The problem may be improved.<ol style="list-style-type: none"><li>1. Press  to turn off the monitor.</li><li>2. Press  holding <b>ENTER</b> down to turn on the monitor.</li></ol>"x" is displayed on the menu title of "Information" of the Setting menu.</li><li>3. Reboot the PC.</li></ul> To return to the previous setting, perform steps 1 to 3 again.				

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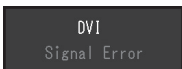
### When Using as a Touch Panel Monitor

- **Windows 8 / Windows 7:** Use the standard Windows touch panel driver. Installation of a new driver is not required. Configure touch panel settings and perform calibration. For details, refer to “Chapter 4 Touch Panel Settings” in the Monitor User’s Manual (on the CD-ROM).  
Use the touch panel driver included on the EIZO LCD Utility Disk (CD-ROM). Refer to the Touch Panel Driver User’s Manual (on the CD-ROM) for installing and configuring the driver.  
**Attention** • Change “Configuration File” to “Mouse.mfd” under “Basic Setting” in the “Touch Screen Properties”. The touch position may not be correctly recognized if the configuration file is “Touch.mfd” (default setting).

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