# User's Manual

## RadiForce® GX540-CL

Monochrome LCD Monitor

### Important

Please read PRECAUTIONS, this User's Manual, and Setup Manual (separate volume) carefully to familiarize yourself with safe and effective usage.

- Please refer to the Setup Manual for basic information ranging from connection of the monitor to a PC to using the monitor.
- The latest User's Manual is available for download from our web site:  
<http://www.eizo.com>

This product has been adjusted specifically for use in the region to which it was originally shipped. If operated outside this region, the product may not perform as stated in the specifications.

---

No part of this manual may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, or otherwise, without the prior written permission of EIZO Corporation.

EIZO Corporation is under no obligation to hold any submitted material or information confidential unless prior arrangements are made pursuant to EIZO Corporation's receipt of said information. Although every effort has been made to ensure that this manual provides up-to-date information, please note that EIZO monitor specifications are subject to change without notice.

---

# Notice for this monitor

---

This product is intended to be used in displaying and viewing digital images including those of digital mammography for review and analysis by trained medical practitioners.

**Caution:** Federal law (USA) restricts this device to sale by or on the order of a physician.

---

This product has been adjusted specifically for use in the region to which it was originally shipped. If the product is used outside the region, it may not operate as specified in the specifications.

---

This product may not be covered by warranty for uses other than those described in this manual.

---

The specifications noted in this manual are only applicable when the following are used:

- Power cords provided with the product
  - Signal cables specified by us
- 

Only use optional products manufactured or specified by us with this product.

---

It takes about 30 minutes for the performance of electrical parts to stabilize. Please wait 30 minutes or more after the monitor power has been turned on or the monitor has recovered from the power saving mode, and then adjust the monitor.

---

It takes about a few minutes for the image quality to reach acceptable level. Please wait a few minutes or more after the monitor power has been turned on or the monitor has recovered from the power saving mode, and then view images for diagnosis.

---

Monitors should be set to a lower brightness to reduce changes in luminosity caused by long-term use and maintain a stable display.

---

When the screen image is changed after displaying the same image for extended periods of time, an afterimage may appear. Use the screen saver or power save function to avoid displaying the same image for extended periods of time.

---

Periodic cleaning is recommended to keep the monitor looking new and to prolong its operation lifetime (refer to “[Cleaning](#)” (page 4)).

---

The screen may have defective pixels or a small number of light dots on the screen. This is due to the characteristics of the panel itself, and is not a malfunction of the product.

---

The backlight of the LCD panel has a fixed lifetime. When the screen becomes dark or begins to flicker, please contact your local EIZO representative.

---

Do not press on the panel or edge of the frame strongly, as this may result in display malfunctions, such as interference patterns, etc. If pressure is continually applied to the panel, it may deteriorate or damage your panel. (If the pressure marks remain on the panel, leave the monitor with a black or white screen. The symptom may disappear.)

---

Do not scratch or press on the panel with any sharp objects, as this may result in damage to the panel. Do not attempt to brush with tissues as this may scratch the panel.

---

When the monitor is cold and brought into a room or the room temperature goes up quickly, dew condensation may occur on the interior and exterior surfaces of the monitor. In that case, do not turn the monitor on. Instead wait until the dew condensation disappears, otherwise it may cause some damage to the monitor.

---

# Cleaning

---

## Attention

- Chemicals such as alcohol and antiseptic solution may cause gloss variation, tarnishing, and fading of the cabinet or panel, and also quality deterioration of the image.
  - Never use any thinner, benzene, wax, and abrasive cleaner, which may damage the cabinet or panel.
- 

## Note

- The optional ScreenCleaner is recommended for cleaning the cabinet and panel surface.
- 

The stains on the cabinet and panel surface can be removed by moistening part of a soft cloth with water.

# To use the monitor comfortably

---

- An excessively dark or bright screen may affect your eyes. Adjust the brightness of the monitor according to the environmental conditions.
- Staring at the monitor for a long time tires your eyes. Take a 10-minute rest every hour.

# CONTENTS

<b>Cover .....</b>	<b>1</b>
<b>Notice for this monitor .....</b>	<b>3</b>
Cleaning .....	4
To use the monitor comfortably .....	4
<b>CONTENTS.....</b>	<b>5</b>
<b>Chapter 1 Introduction.....</b>	<b>7</b>
<b>1-1. Features .....</b>	<b>7</b>
<b>1-2. EIZO LCD Utility Disk.....</b>	<b>8</b>
● Disk contents and software overview .....	8
● Using RadiCS LE/ScreenManager Pro for Medical.....	8
<b>1-3. Basic Operations and Functions .....</b>	<b>9</b>
● Basic Adjustment menu operations .....	9
● Functions .....	10
<b>Chapter 2 Adjusting Screens .....</b>	<b>12</b>
<b>2-1. Compatible Resolutions .....</b>	<b>12</b>
<b>2-2. Setting the Resolution .....</b>	<b>12</b>
● Windows 8 / Windows 7 .....	12
● Windows Vista.....	12
● Windows XP .....	12
● Mac OS X .....	12
<b>2-3. Adjusting Mode.....</b>	<b>13</b>
● Selecting the display mode (CAL Switch mode) .....	13
● Adjustable settings in each mode .....	14
● Adjusting Brightness .....	14
● Setting the mode in the monochrome image display area .....	15
● Setting the mode in the base display area.....	15
● Setting the detection type in the image display area .....	16
● Setting the highlight area in the image display area .....	16
● Setting the sensitivity level for detecting the ambient illuminance .....	16
● Setting ambient illuminance and related settings for Lamb.....	17
● Displaying the current value of the ambient luminance .....	17
<b>Chapter 3 Setting Monitor .....</b>	<b>18</b>
<b>3-1. Setting Orientation .....</b>	<b>18</b>
<b>3-2. Setting Input Signal .....</b>	<b>19</b>
● To connect DVI.....	19
● To connect DisplayPort .....	19
<b>3-3. Enabling/Disabling Mode Selection .....</b>	<b>20</b>
<b>3-4. Setting Language.....</b>	<b>20</b>
<b>3-5. Setting Adjustment Menu Display .....</b>	<b>20</b>
● Changing adjustment menu display position .....	20
● Showing and hiding the EIZO logo .....	21
● Locking Operation Buttons .....	21
● Enabling/Disabling DDC Communication .....	22
● Enabling/Disabling DC5V Output.....	22
<b>3-10. Resetting to Default.....</b>	<b>23</b>
● Resetting the mode adjustments .....	23
● Resetting all adjustments to the default settings .....	23
<b>Chapter 4 Controlling Monitor Quality .....</b>	<b>24</b>
<b>4-1. Executing Calibration.....</b>	<b>24</b>
● Executing SelfCalibration.....	24
● Viewing SelfCalibration result .....	25
<b>4-2. Checking Grayscale.....</b>	<b>25</b>
● Executing Grayscale Check .....	25
● Viewing grayscale check results .....	25
<b>4-3. Acquiring Life Prediction Data .....</b>	<b>26</b>
<b>4-4. Setting On/Off for Warning and QC             History.....</b>	<b>26</b>
<b>4-5. Using Quick Image Checker function.....</b>	<b>26</b>
<b>Chapter 5 Power Saving Functions .....</b>	<b>27</b>
<b>5-1. Setting Power Save.....</b>	<b>27</b>
<b>5-2. Adjusting the Brightness of Indicator.....</b>	<b>27</b>
<b>5-3. Setting DisplayPort Power Save .....</b>	<b>28</b>
<b>5-4. Setting the Power Saving Mode when a             Person Leaves the Monitor .....</b>	<b>29</b>
<b>Chapter 6 Troubleshooting.....</b>	<b>30</b>
<b>6-1. No Picture .....</b>	<b>30</b>
<b>6-2. Imaging Problems .....</b>	<b>31</b>
<b>6-3. Other Problems .....</b>	<b>32</b>
● Error code table.....	33
<b>Chapter 7 Reference .....</b>	<b>34</b>
<b>7-1. Attaching Arm .....</b>	<b>34</b>
<b>7-2. Connecting Multiple PCs .....</b>	<b>35</b>
● Switching among input signals.....	35
● Setting input signal switching procedure .....	36
<b>7-3. Making Use of USB (Universal Serial             Bus).....</b>	<b>37</b>
● Required system environment .....	37
● Connection procedure (Setup of USB function) .....	37
<b>7-4. Showing Monitor Information .....</b>	<b>38</b>
● Displaying signal information .....	38
● Showing monitor information .....	38
<b>7-5. Specifications .....</b>	<b>39</b>
<b>Chapter 8 Glossary .....</b>	<b>42</b>

<b>Appendix .....</b>	<b>43</b>
Trademark .....	43
License .....	43
Medical Standard .....	44
FCC Declaration of Conformity .....	45
EMC Information .....	46
LIMITED WARRANTY.....	49
Recycling Information .....	56
China Measures for RoHS Compliance in ChinaRoHS.....	60

# Chapter 1 Introduction

Thank you very much for choosing an EIZO monochrome LCD monitor.

## 1-1. Features

- 21.3 inches
- Supports a resolution of 5M pixels (Portrait: 2048 × 2560 dots (H × V))
- Uses high-contrast panel (1200:1).  
Enables the display of sharp images.
- Applicable to DisplayPort (applicable to 8 bit or 10 bit, not applicable to audio signals)
- The Hybrid Gamma function automatically identifies the display area for the medical and other images on the same screen, and displays each set status.  
\*1 The identification may fail depending on the displayed image. The software to be used needs to be verified. For validation, see “[Setting the highlight area in the image display area](#)” (page 16).
- The RadiCS SelfQC function and the built-in integrated front sensor allow the user to perform the calibration and Grayscale Check for the monitor independently.  
See “[Chapter 4 Controlling Monitor Quality](#)” (page 24).
- \*2 When a panel protector (RP-901) is installed, the Integrated Front Sensor cannot be used.
- The CAL Switch function allows the user to select the display mode optimum to the displayed image.
  - The ALT mode adjusts the brightness according to the ambient illuminance.
  - Most suitable mode for calibration  
See “[Selecting the display mode \(CAL Switch mode\)](#)” (page 13).
- Selectable DICOM ([page 42](#)) Part 14 complied screen.
- The quality control software “RadiCS LE” used to calibrate the monitor and to manage the history is included.  
See “[1-2. EIZO LCD Utility Disk](#)” (page 8).
- The software “ScreenManager Pro for Medical” to adjust the screen using the mouse and keyboard is included  
See “[1-2. EIZO LCD Utility Disk](#)” (page 8).
- Frame Synchronous mode supported (24.5 to 25.5 Hz (DVI signal input only), 49 to 51 Hz)
- Power saving function  
This product is equipped with power saving function.
  - 0 W power consumption when the main power is off  
Equipped with main power switch.  
When the monitor is not required, the power supply can be shut off using the main power switch
  - Presence Sensor  
The sensor on the front side of the monitor detects the movement of a person. When a person moves away from the monitor, the monitor shifts to the power saving mode automatically and does not display the images on the screen. Therefore, the function reduces the power consumption.  
See “[5-4. Setting the Power Saving Mode when a Person Leaves the Monitor](#)” (page 29).
- Long service life LED backlight LCD panel

- Built-in ambient light sensor

The ambient light sensor can measure the ambient illuminance in lux. Depending on the environment, the sensor sometimes indicates different values from those measured by a stand-alone illuminance meter.

The built-in ambient light sensor is used in the ALT mode or when the measurement is triggered by user operation in the corresponding menu of the “RadiCS/RadiCS LE” quality control software.

For further details about the measured value and how to execute the measurement, refer to the RadiCS/RadiCS LE User’s Manual (on the CD-ROM).

- Displays HDCP (High-bandwidth Digital Content Protection) protected contents (DisplayPort only).

#### **Attention**

- A high temperature or high humidity environment may affect the measurement accuracy of the integrated front sensor. We suggest storing and using the monitor under the following conditions.
  - Temperature 30°C or less
  - Humidity 70% or less
- Avoid storing or using the sensor where it may be exposed to direct sunlight.

## 1-2. EIZO LCD Utility Disk

An “EIZO LCD Utility Disk” (CD-ROM) is supplied with this product. The following table shows the disk contents and the overview of the software programs.

### ● Disk contents and software overview

The disk includes application software programs for adjustment, and User’s Manual. Refer to Readme.txt file on the disk for software startup procedures or file access procedures.

Contents	Overview
Readme.txt file	
RadiCS LE (for Windows)	Quality control software for calibrating the monitor and managing the calibration history.
ScreenManager Pro for Medical (for Windows)	Software for adjusting the screen using the mouse and keyboard.
User’s Manual of this monitor (PDF file)	

### ● Using RadiCS LE/ScreenManager Pro for Medical

For the installation and use of “RadiCS LE/ScreenManager Pro for Medical”, refer to the respective User’s Manual on the disk.

When using this software, you will need to connect a PC to the monitor with the supplied USB cable. For more information, see “[7-3. Making Use of USB \(Universal Serial Bus\)](#)” (page 37).

# 1-3. Basic Operations and Functions

## ● Basic Adjustment menu operations

### 1. Displaying Adjustment menu

1. Press ENTER. The Adjustment menu appears.

### 2. Adjusting/Setting

1. Select a menu to adjust/set with ▲ or ▼, and then press ENTER.

2. Select an item to adjust/set with ▲ or ▼, and then press ENTER.

3. Adjust/set the selected item with ▲ or ▼, and then press ENTER.

### 3. Exiting the software

1. Press RETURN a few times. The Adjustment menu finishes.

## ● Functions

The following table shows the Adjustment menu and setting items of each menu.

Main Menu	Item	Reference
Mode	 Brightness  Gray Area  Base Area  Hybrid Assignment  Highlight Area  Sensitivity  Lamb Target (Low)  Lamb Target (Std.)  Lamb Target (High)  Current Status  Mode Reset	<a href="#">“2-3. Adjusting Mode” (page 13)</a> <a href="#">“3-10. Resetting to Default” (page 23)</a>
RadiCS SelfQC	 SelfCalibration  Grayscale Check  LEA  Settings	<a href="#">“4-1. Executing Calibration” (page 24)</a> <a href="#">“4-2. Checking Grayscale” (page 25)</a> <a href="#">“4-3. Acquiring Life Prediction Data” (page 26)</a> <a href="#">“4-4. Setting On/Off for Warning and QC History” (page 26)</a>
Power Manager	 Power Save  Indicator  Presence Sensor	<a href="#">“5-1. Setting Power Save” (page 27)</a> <a href="#">“5-2. Adjusting the Brightness of Indicator” (page 27)</a> <a href="#">“5-4. Setting the Power Saving Mode when a Person Leaves the Monitor” (page 29)</a>
Menu Settings	 Language  Menu Position	<a href="#">“3-4. Setting Language” (page 20)</a> <a href="#">“3-5. Setting Adjustment Menu Display” (page 20)</a>
Tools	 Input  Input Selection  Mode Preset  Signal Info  Monitor Info  All Reset	<a href="#">“Switching among input signals” (page 35)</a> <a href="#">“Setting input signal switching procedure” (page 36)</a> <a href="#">“3-3. Enabling/Disabling Mode Selection” (page 20)</a> <a href="#">“7-4. Showing Monitor Information” (page 38)</a> <a href="#">“3-10. Resetting to Default” (page 23)</a>

Main Menu	Item		Reference
Optional Settings	Logo		"3-6. Showing and hiding the EIZO logo" (page 21)
	Key Lock		"3-7. Locking Operation Buttons" (page 21)
	Orientation		"3-1. Setting Orientation" (page 18)
	Input Signal		"3-2. Setting Input Signal" (page 19)
	Connection Config.	DDC	"3-8. Enabling/Disabling DDC Communication" (page 22)
		DC5V Output	"3-9. Enabling/Disabling DC5V Output" (page 22)
		DP Power Save	"5-3. Setting DisplayPort Power Save" (page 28)

# Chapter 2 Adjusting Screens

## 2-1. Compatible Resolutions

For details on compatible resolutions, refer to “Compatible Resolutions” in the Setup Manual.

## 2-2. Setting the Resolution

### ● Windows 8 / Windows 7

1. For Windows 8, click the “Desktop” tile on the Start Screen to display the desktop.
2. Right-click the mouse anywhere on the desktop except for icons.
3. From the displayed menu, click “Screen resolution”.
4. On the “Screen Resolution” dialog box, select the monitor.
5. Click “Resolution” to select the desired resolution.
6. Click the “OK” button.
7. When a confirmation dialog box is displayed, click “Keep changes”.

### ● Windows Vista

1. Right-click the mouse anywhere on the desktop except for icons.
2. From the displayed menu, click “Personalize”.
3. On the “Personalization” window, click “Display Settings”.
4. On the “Display Settings” dialog box, select the “Monitor” tab and select desired resolution in the “Resolution” field.
5. Click the “OK” button.
6. When a confirmation dialog box is displayed, click “Yes”.

### ● Windows XP

1. Right-click the mouse anywhere on the desktop except for icons.
2. From the displayed menu, click “Properties”.
3. When the “Display Properties” dialog box is displayed, click the “Settings” tab and select desired resolution for “Screen resolution” under “Display”.
4. Click the “OK” button to close the dialog box.

### ● Mac OS X

1. Select “System Preferences” from the Apple menu.
2. When the “System Preferences” dialog box is displayed, click “Displays” for “Hardware”.
3. On the displayed dialog box, select the “Display” tab and select desired resolution in the “Resolutions” field.
4. Your selection will be reflected immediately. When you are satisfied with the selected resolution, close the window.

## 2-3. Adjusting Mode

### ● Selecting the display mode (CAL Switch mode)


A desired mode can be selected easily according to the monitor application.

#### Display Modes

√: Calibration is available.

Mode	Purpose	
1-DICOM	Selected for the display in the DICOM mode.	√
2-CAL1 / 3-CAL2	Displays the screen adjusted by calibration software.	√
4-Hybrid-γ	Automatically identifies the display area for the medical and other images on the same screen, and displays each set status.	-
5-ALT <sup>*1</sup>	Suitable for displaying the images with the brightness and grayscale appropriate for the ambient illuminance.	-

\*1 The screen may not display properly depending on the usage environment. The quality of the monitor cannot be managed using this mode.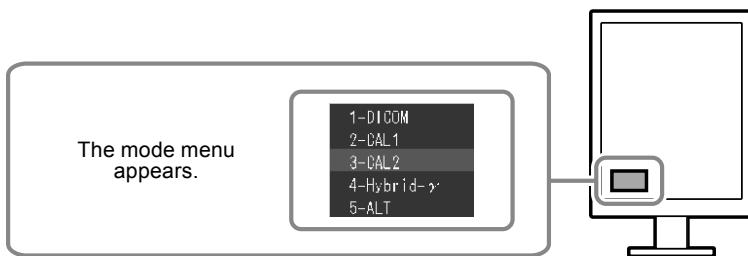

#### Procedure

1. Press MODE.  
The mode menu appears at the bottom left of the screen.
2. The mode among the list is highlighted in turn each time MODE is pressed.  
You can switch the mode with ▲ or ▼ while the mode menu is displayed.

→1-DICOM←→2-CAL1←→3-CAL2←→4-Hybrid-γ←→5-ALT←→

#### Note

- The Adjustment menu and the Mode menu cannot be displayed at the same time.

## ● Adjustable settings in each mode

The adjustable settings depend on the mode. (Non-adjustable or non-settable function cannot be selected.)

√: Adjustable -: Not adjustable

Icon	Function	CAL Switch mode			
		1-DICOM	2-CAL1 3-CAL2	4-Hybrid-γ	5-ALT
	Brightness	√	√	-	-
	Gray Area	-	-	√	-
	Base Area	-	-	√	-
	Hybrid Assignment	-	-	√	-
	Highlight Area	-	-	√	-
	Sensitivity	-	-	-	√
	Lamb Target (Low)	-	-	-	√
	Lamb Target (Std.)	-	-	-	√
	Lamb Target (High)	-	-	-	√
	Mode Reset	√	√	√	√

## ● Adjusting Brightness

The screen brightness is adjusted by changing the brightness of the backlight (light source from the LCD back panel).

### Setting Range

0 to 100%

### Procedure

1. Select “Mode” from the Adjustment menu, and press ENTER.
2. Select “Brightness” from “Mode”, and press ENTER.
3. Use ▲ or ▼ for adjustment.
4. Press ENTER to exit.

## ● Setting the mode in the monochrome image display area

The mode suitable for the monochrome image display area can be set (see “[Selecting the display mode \(CAL Switch mode\)](#)” (page 13)).

### Setting Range

DICOM, CAL1, CAL2

### Procedure

1. Select “Mode” from the Adjustment menu, and press ENTER.
2. Select “Gray Area” from “Mode”, and press ENTER.
3. Use ▲ or ▼ for adjustment.
4. Press ENTER to exit.

---

#### Attention

- After the configuration in the display area, you cannot select any mode for which Mode Preset is “Off” (see “[3-3. Enabling/Disabling Mode Selection](#)” (page 20)).

---

#### Note

- The brightness is the same as the value for the mode selected. To change the brightness, use the mode selected for adjustment.

## ● Setting the mode in the base display area

The mode suitable for the base display area can be set.

### Setting Range

Base-L, Base-H

### Procedure

1. Select “Mode” from the Adjustment menu, and press ENTER.
2. Select “Base Area” from “Mode”, and press ENTER.
3. Select “Base-L” or “Base-H” with ▲ or ▼.

Mode	Purpose
Base-L	Specifies the target brightness to 100 cd/m <sup>2</sup> .
Base-H	Specifies the target brightness to 170 cd/m <sup>2</sup> .

4. Press ENTER to exit.

---

#### Attention

- After the configuration in the display area, you cannot select any mode for which Mode Preset is “Off” (see “[3-3. Enabling/Disabling Mode Selection](#)” (page 20)).

---

#### Note

- If the brightness value in the base display area is higher than the target brightness, the target brightness is applied for Base-L and Base-H.

## ● Setting the detection type in the image display area

The detection type in the display area can be set.

### Procedure

1. Select “Mode” from the Adjustment menu, and press ENTER.
2. Select “Hybrid Assignment” from “Mode”, and press ENTER.
3. Select “Full Scan”, “Partial Scan”, or “Custom” with ▲ or ▼.

Detection Type	Purpose
Full Scan	Identifies the display area automatically.
Partial Scan	Identifies the selection area using software (Hybrid Gamma Manager) automatically.
Custom	Allows you to set the display area manually using software (Hybrid Gamma Manager).

4. Press ENTER to exit.

### Note

- For Hybrid Gamma Manager, contact your local EIZO representative.

## ● Setting the highlight area in the image display area

On/Off for the highlight area in the image display area can be set.

### Procedure

1. Select “Mode” from the Adjustment menu, and press ENTER.
2. Select “Highlight Area” from “Mode”, and press ENTER.
3. Select “On” or “Off” with ▲ or ▼.
4. Press ENTER to exit.

### Note

- You can set On/Off for the highlight area with ▼.

## ● Setting the sensitivity level for detecting the ambient illuminance

The sensitivity level for detecting the ambient illuminance (running mean time) can be adjusted. The higher the sensitivity level is, the more frequently the brightness and LUT will be updated.

### Setting Range

Level 1 to level 5

### Procedure

1. Select “Mode” from the Adjustment menu, and press ENTER.
2. Select “Sensitivity” from “Mode”, and press ENTER.
3. Use ▲ or ▼ for adjustment.
4. Press ENTER to exit.

## ● Setting ambient illuminance and related settings for Lamb

In the ALT mode, the brightness and LUT of the monitor display are changed according to the desired target value specified for the ambient luminance (Lamb). Three types of settings are available according to your environment.

Item	Purpose
Lamb Target (Std.)	The illuminance in the standard environment can be selected. Select the desired minimum luminance, and maximum luminance suitable for the illuminance.
Lamb Target (Low)	The illuminance in the darkest environment can be selected. Select the desired minimum luminance, and maximum luminance suitable for the illuminance.
Lamb Target (High)	The illuminance in the brightest environment can be selected. Select the desired minimum luminance, and maximum luminance suitable for the illuminance.

### Illuminance (Ix)

#### Procedure

1. Select “Mode” from the Adjustment menu, and press ENTER.
2. Select “Lamb Target (Low)”, “Lamb Target (Std.)”, or “Lamb Target (High)” from “Mode”, and press ENTER.
3. Select “Illuminance”, and press ENTER.
4. Use ▲ or ▼ for adjustment.
5. Press ENTER to exit.

### Minimum brightness value/maximum brightness value

#### Procedure

1. Select “Mode” from the Adjustment menu, and press ENTER.
2. Select “Lamb Target (Low)”, “Lamb Target (Std.)”, or “Lamb Target (High)” from “Mode”, and press ENTER.
3. Select “Lmin” or “Lmax”, and press ENTER.
4. Use ▲ or ▼ for adjustment.
5. Press ENTER to exit.

## ● Displaying the current value of the ambient luminance

The current value of the ambient luminance can be displayed.

#### Procedure

1. Select “Mode” from the Adjustment menu, and press ENTER.
2. Select “Current Status” from “Mode”, and press ENTER.

The current values of the illuminance, minimum luminance, maximum luminance, and ambient luminance are displayed.

# Chapter 3 Setting Monitor

## 3-1. Setting Orientation



The display direction of the image can be changed according to your environment.

### Attention

- Refer to the User's Manual of the graphics board.
- The setting is available even when the PC is not started up.

### Procedure

1. Press **⊕** to turn off the monitor.
2. Hold down MODE while pressing **⊕** for at least two seconds to turn on the monitor.  
The “Optional Settings” menu appears.
3. Select “Orientation” from “Optional Settings”, and press ENTER.
4. Select “Landscape”, “Portrait(SW)”, or “Portrait(HW)” with **▲** or **▼**, and press ENTER.

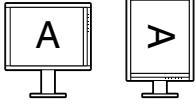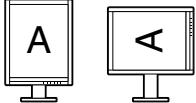Monitor orientation	Display example
Landscape Selected for horizontal installation.	
Portrait(SW) Selected for vertical installation. The display image is turned by 90° using the utility software of the graphics board.	
Portrait(HW) Selected for vertical installation. The display image is turned by 90° using the monitor function.	

5. Select “Finish” with **▲** or **▼**.
6. Press ENTER.

The “Optional Settings” menu is closed.

## 3-2. Setting Input Signal

---

The settings for the input signal may be required depending on the usage environment. Signals can be selected based on your operating environment.

### **Attention**

- A supported graphics board is required to display.
  - Refer to the User's Manual of the graphics board.
  - Setting is available even when the PC is not started up.
- 

### ● To connect DVI

#### **Procedure**

1. Press **⊕** to turn off the monitor.
2. Hold down **MODE** while pressing **⊕** for at least two seconds to turn on the monitor.  
The “Optional Settings” menu appears.
3. Select “Signal Selection” from “Optional Settings”, and press **ENTER**.
4. Select “DVI” with **▲** or **▼**, and press **ENTER**.
5. Select “Signal Format” with **▲** or **▼**, and press **ENTER**.
6. Select “Standard” or “SDG 8” with **▲** or **▼**, and press **ENTER**.
7. Select “Sub Pixel Drive” with **▲** or **▼**, and press **ENTER**.
8. Leave the default setting “Off” as it is, and press **ENTER**.
9. Press **RETURN** twice.
10. Select “Finish” with **▲** or **▼**.
11. Press **ENTER**.

The “Optional Settings” menu is closed.

### **Attention**

- If “Sub Pixel Drive” is set to “On”, the screen is not displayed properly.
- 

### ● To connect DisplayPort

#### **Procedure**

1. Press **⊕** to turn off the monitor.
2. Hold down **MODE** while pressing **⊕** for at least two seconds to turn on the monitor.  
The “Optional Settings” menu appears.
3. Select “Signal Selection” from “Optional Settings”, and press **ENTER**.
4. Select “DisplayPort” with **▲** or **▼**, and press **ENTER**.
5. Select “Sub Pixel Drive” with **▲** or **▼**, and press **ENTER**.
6. Leave the default setting “Off” as it is, and press **ENTER**.
7. Press **RETURN** twice.
8. Select “Finish” with **▲** or **▼**.
9. Press **ENTER**.

The “Optional Settings” menu is closed.

### **Attention**

- If “Sub Pixel Drive” is set to “On”, the screen is not displayed properly.
-

## 3-3. Enabling/Disabling Mode Selection

Only the modes currently being set can be selected.

This setting is suitable when you are using limited display modes and you do not want to change the customized display state.

### Procedure

1. Select “Tools” from the Adjustment menu, and press ENTER.
2. Select “Mode Preset” from “Tools”, and press ENTER.
3. Use ▲ or ▼ to select the mode to change, and press ENTER.
4. Select “On” or “Off” with ▲ or ▼.
5. Press ENTER to exit.

### Attention

- You cannot set all the modes disable. Set at least one mode to “On”.

## 3-4. Setting Language

The language for the Setting menu and messages can be selected.

### Selectable languages

English/German/French/Spanish/Italian/Swedish/Japanese/Simplified Chinese/Traditional Chinese

### Procedure

1. Select “Menu Settings” from the Adjustment menu, and press ENTER.
2. Select “Language” from “Menu Settings”, and press ENTER.
3. Select a language with ▲ or ▼.
4. Press ENTER to exit.

## 3-5. Setting Adjustment Menu Display

### ● Changing adjustment menu display position

You can change the display position of the Adjustment menu.

### Procedure

1. Select “Menu Settings” from the Adjustment menu, and press ENTER.
2. Select “Menu Position” from “Menu Settings”, and press ENTER.
3. Select the position with ▲ or ▼.
4. Press ENTER to exit.

## 3-6. Showing and hiding the EIZO logo

You can choose to show or hide the display of the EIZO logo.

### **Procedure**

1. Press  $\text{P}$  to turn off the monitor.
2. Hold down **MODE** while pressing  $\text{P}$  for at least two seconds to turn on the monitor.  
The “Optional Settings” menu appears.
3. Select “Logo” from “Optional Settings”, and press **ENTER**.
4. Select “On” or “Off” with  $\blacktriangle$  or  $\blacktriangledown$ , and press **ENTER**.
5. Select “Finish” with  $\blacktriangle$  or  $\blacktriangledown$ .
6. Press **ENTER**.  
The “Optional Settings” menu is closed.

## 3-7. Locking Operation Buttons

The settings can be locked to prevent them being changed.

### **Procedure**

1. Press  $\text{P}$  to turn off the monitor.
2. Hold down **MODE** while pressing  $\text{P}$  for at least two seconds to turn on the monitor.  
The “Optional Settings” menu appears.
3. Select “Key Lock” from “Optional Settings”, and press **ENTER**.
4. Select “Off”, “Menu”, or “All” with  $\blacktriangle$  or  $\blacktriangledown$ , and press **ENTER**.

Item	Buttons that can be locked
Off (Default setting)	None (All buttons are enabled)
Menu	<b>ENTER</b> button
All	All buttons excluding $\text{P}$

5. Select “Finish” with  $\blacktriangle$  or  $\blacktriangledown$ .
6. Press **ENTER**.  
The “Optional Settings” menu is closed.

## 3-8. Enabling/Disabling DDC Communication

The DDC communication can be enabled/disabled. Turn on this setting normally. (This is different from the DDC/CI communication.)

### **Procedure**

1. Press **⊕** to turn off the monitor.
2. Hold down **MODE** while pressing **⊕** for at least two seconds to turn on the monitor.  
The “Optional Settings” menu appears.
3. Select “Connection Config.” from “Optional Settings”, and press **ENTER**.
4. Select “DDC” with **▲** or **▼**, and press **ENTER**.
5. Select “On” or “Off” with **▲** or **▼**, and press **ENTER**.
6. Press **RETURN** to exit.
7. Select “Finish” with **▲** or **▼**.
8. Press **ENTER**.

The “Optional Settings” menu is closed.

## 3-9. Enabling/Disabling DC5V Output

The DC5V output can be enabled/disabled. Turn off this setting normally.

### **Procedure**

1. Press **⊕** to turn off the monitor.
2. Hold down **MODE** while pressing **⊕** for at least two seconds to turn on the monitor.  
The “Optional Settings” menu appears.
3. Select “Connection Config.” from “Optional Settings”, and press **ENTER**.
4. Select “DC5V Output” with **▲** or **▼**, and press **ENTER**.
5. Select “On” or “Off” with **▲** or **▼**, and press **ENTER**.
6. Press **RETURN** to exit.
7. Select “Finish” with **▲** or **▼**.
8. Press **ENTER**.

The “Optional Settings” menu is closed.

## 3-10. Resetting to Default

---

There are two different Resets: Mode Reset for only resetting the mode adjustments to the default settings and All Reset for resetting all settings to the defaults.

### Attention

- After resetting, you cannot undo the operation.

### Note

- Regarding the default setting, see “[Main Default Settings](#)” (page 40).

### ● Resetting the mode adjustments

Only the mode adjustment value for the currently selected mode can be reset to the default settings.

#### Procedure

1. Select “Mode” from the Adjustment menu, and press ENTER.
2. Select “Mode Reset” from “Mode”, and press ENTER.
3. Select “Execute” with ▲ or ▼.
4. Press ENTER.

The mode adjustments revert to the default settings.

### ● Resetting all adjustments to the default settings

All settings are reset to the default settings (except the “Input” and “Optional Settings” menus).

#### Procedure

1. Select “Tools” from the Adjustment menu, and press ENTER.
2. Select “All Reset” from “Tools”, and press ENTER.
3. Select “Execute” with ▲ or ▼.
4. Press ENTER.

All settings excluding the “Input” and “Optional Settings” menus are reset to the default settings.

# Chapter 4 Controlling Monitor Quality

This product has the built-in Integrated Front Sensor, which allows the user to perform periodic calibration and Grayscale Check for the monitor independently.

The pattern images for checking the quality of the monitor are loaded on the monitor so that the patterns can be checked with an independent monitor.

## Attention

- When a panel protector (RP-901) is installed, the Integrated Front Sensor cannot be used. Do not execute “SelfCalibration” or “Grayscale Check” of “RadiCS SelfQC”. Be sure to set “Meas. Frequency” to “Off”.
- When a panel protector (RP-911) is installed, the Integrated Front Sensor can be used. After installation, be sure to execute correlation using RadiCS/RadiCS LE.

## Note

- To perform advanced quality control that conforms to medical standards, use the optional monitor quality control software kit “RadiCS UX1”.
- To align the measurement result of the integrated front sensor with that of the UX1 sensor to be set as a reference, execute correlation using RadiCS/RadiCS LE. Periodical correlation enables the measurement accuracy of the integrated front sensor equivalent to that of the UX1 sensor. For details, refer to RadiCS/RadiCS LE User’s Manual.
- The adjustment target for the calibration, judgment value for the grayscale check, and schedule for both of them can be set using the “RadiCS” software.
- The adjustment target and schedule for the calibration can be set using the supplied “RadiCS LE” software. For details, refer to RadiCS LE User’s Manual (on the CD-ROM).
- After changing the orientation of the monitor, be sure to perform correlation.

## 4-1. Executing Calibration

SelfCalibration can be executed and latest result can be viewed. In addition, when the schedule is set using RadiCS/RadiCS LE, the next calibration schedule appears.

### ● Executing SelfCalibration

#### Attention

- Do not execute this function when a panel protector (RP-901) is installed.
- After SelfCalibration is executed, execute Grayscale Check under the actual temperature of the monitor usage environment.

#### Procedure

1. Select “RadiCS SelfQC” from the Adjustment menu, and press ENTER.
2. Select “SelfCalibration” from “RadiCS SelfQC”, and press ENTER.
3. Select “Execute” with ▲ or ▼.
4. Press ENTER to exit.

SelfCalibration is executed.

## ● Viewing SelfCalibration result

### Attention

- The result can be viewed when “QC History” is “On” (see “[4-4. Setting On/Off for Warning and QC History](#)” (page 26)).

### Procedure

1. Select “RadiCS SelfQC” from the Adjustment menu, and press ENTER.
2. Select “SelfCalibration” from “RadiCS SelfQC”, and press ENTER.
3. Select “Result” with ▲ or ▼.
4. Select “DICOM”, “CAL1” or “CAL2” with ▲ or ▼.
5. Press ENTER to exit.

The SelfCalibration result (Max. Error Rate, Last Time, and Error Code) appears.

## 4-2. Checking Grayscale

Grayscale Check can be executed and the latest result can be viewed. In addition, when the schedule is set using RadiCS, the next grayscale check schedule appears.

## ● Executing Grayscale Check

### Attention

- Do not execute this function when a panel protector (RP-901) is installed.
- Execute Grayscale Check under the actual temperature of the monitor usage environment.

### Procedure

1. Select “RadiCS SelfQC” from the Adjustment menu, and press ENTER.
2. Select “Grayscale Check” from “RadiCS SelfQC”, and press ENTER.
3. Select “Execute” with ▲ or ▼.
4. Press ENTER to exit.

Grayscale Check is executed.

## ● Viewing grayscale check results

### Attention

- The results can be viewed when “QC History” is “On”.

### Procedure

1. Select “RadiCS SelfQC” from the Adjustment menu, and press ENTER.
2. Select “Grayscale Check” from “RadiCS SelfQC”, and press ENTER.
3. Select “Result” with ▲ or ▼.
4. Select “DICOM”, “CAL1” or “CAL2” with ▲ or ▼.
5. Press ENTER to exit.

Up to the five latest grayscale check results (Passed/Failed/Canceled/ERR) appear.

## 4-3. Acquiring Life Prediction Data

The data necessary to predict the monitor life can be acquired.

### Attention

- When a panel protector (RP-901) is installed, be sure to set “Meas. Frequency” to “Off”.

### Note

- Data are collected every 100 hours.
- When you select “Power Save”, data are collected when the mode switches to Power Save or the secondary power off after the above timing is reached.
- When you select “Routine”, data are saved after the following times have elapsed.
  - 500, 1000, 2000, 4000, 7000, 10000, 15000, 20000, 25000, 30000, 35000, 40000, 45000, 50000 hours

### Procedure

- Select “RadiCS SelfQC” from the Adjustment menu, and press ENTER.
- Select “LEA” from “RadiCS SelfQC”, and press ENTER.
- Select “Meas. Frequency” with ▲ or ▼.
- Select “Power Save”, “Routine”, or “Off” with ▲ or ▼.
- Press ENTER to exit.

## 4-4. Setting On/Off for Warning and QC History

The warning and QC history displays for the failure of the Grayscale Check result can be set to on/off.

### Procedure

- Select “RadiCS SelfQC” from the Adjustment menu, and press ENTER.
- Select “Settings” from “RadiCS SelfQC”, and press ENTER.
- Select “Warning” or “QC History” with ▲ or ▼.
- Select “On” or “Off” with ▲ or ▼.
- Press ENTER to exit.

## 4-5. Using Quick Image Checker function

A test pattern is displayed to enable a simplified check of the monitor display. (Quick Image Checker function)

### Attention

- The Quick Image Checker function is not available under the following conditions.
  - RadiCS/RadiCS LE is running (exit RadiCS/RadiCS LE if it is running).
  - Adjustment menu is displayed.
  - Other than the DICOM/CAL1/CAL2 mode is used.
  - “Sub Pixel Drive” is set to “On”.
- The Quick Image Checker function requires several minutes to prepare the patterns immediately after the monitor power is turned on. Please wait for a while.

### Note

- The AAPM test pattern is saved on the monitor by default.
- Only one pattern can be saved on the monitor.
- To change the pattern image, use RadiCS/RadiCS LE. For details, refer to RadiCS/RadiCS LE User’s Manual.

### Procedure

- Hold down ▲ for two seconds or more.
- The pattern image is displayed.

# Chapter 5 Power Saving Functions

## 5-1. Setting Power Save

The monitor can be set to the power saving mode according to the PC status. When the monitor has shifted to the power saving mode, the images on the screen are not displayed.

### Attention

- Turning off the main power switch or unplugging the power cord completely shuts off power supply to the monitor.
- Devices connected to the USB downstream port work even when the monitor is in power saving mode or when the power button of the monitor is Off.
- Five seconds before the monitor enters the power saving mode, it displays a preliminary message.

### Procedure

1. Select “PowerManager” from the Adjustment menu, and press ENTER.
2. Select “Power Save” from “PowerManager”, and press ENTER.
3. Select “Off” and either “High” or “Low” with ▲ or ▼.

Setting	Function
High	Turns off the backlight. The power saving effect is the greatest.
Low	Turns down the backlight to the lowest setting. The time until the operation stabilizes after exiting the Power Save mode next time can be reduced.

4. Press ENTER to exit.

### Power Saving System

#### At DVI signal input

This monitor complies with the “DVI DMPPM” standard.

#### At DisplayPort signal input

This monitor complies with the “DisplayPort Standard V1.1a”.

The monitor enters the power saving mode after five seconds in connection with the PC setting.

PC	Monitor	Power indicator
On	Operation mode	Green
Power saving mode	Power saving mode	Orange

## 5-2. Adjusting the Brightness of Indicator

The brightness of the power indicator (green) when the screen is displayed can be set. (The power indicator is set to light up when the power is switched on, and set to 4 by default.)

### Procedure

1. Select “PowerManager” from the Adjustment menu, and press ENTER.
2. Select “Indicator” from “PowerManager”, and press ENTER.
3. Select the indicator brightness “Off” or in the range from 1 to 7 with ▲ or ▼ as desired.
4. Press ENTER to exit.

## 5-3. Setting DisplayPort Power Save

---

The power setting of the DisplayPort of the monitor can be switched in the power saving mode.

### Attention

- When “On” is set, the display position on the window may not be positioned correctly when the monitor power is turned on or off. In such a case, turn the setting to “Off”.

### Procedure

1. Press **⊕** to turn off the monitor.
2. Hold down **MODE** while pressing **⊕** for at least two seconds to turn on the monitor.  
The “Optional Settings” menu appears.
3. Select “Connection Config.” from “Optional Settings”, and press **ENTER**.
4. Select “DP Power Save”, and press **ENTER**.
5. Select “On” or “Off” with **▲** or **▼**, and press **ENTER**.
6. Press **RETURN** to exit.
7. Select “Finish” with **▲** or **▼**.
8. Press **ENTER**.

The “Optional Settings” menu is closed.

## 5-4. Setting the Power Saving Mode when a Person Leaves the Monitor

The sensor on the front side of the monitor detects the movement of a person by setting Presence Sensor to “Auto” or “Manual”. When a person moves away from the monitor, it shifts to the power saving mode automatically and does not display the images on the screen to reduce the power consumption. When a person comes near the monitor again, it recovers from the power saving mode and displays the images. When “Manual” is selected, the detection area, sensitivity, and time until the power saving mode is activated can be set according to the monitor usage environment and movement of the user.

### Note

- At the time of shifting to power saving mode, the message that indicates shifting is displayed.

### Attention

- This setting cannot be used during “RadiCS SelfQC” execution.
- It also cannot be used during the landscape position display.

### Procedure



- Select “PowerManager” from the Adjustment menu, and press ENTER.
- Select “Presence Sensor” from “PowerManager”, and press ENTER.
- Select “Auto”, “Manual”, or “Off” with ▲ or ▼.
- Press ENTER to exit.
- After selecting “Manual”, set the details.

Item	Setting Range	Description
Range	70 cm 90 cm 120 cm	If a person leaves the specified detection area (perpendicular to the screen), the monitor shifts to the power saving mode. The setting ranges (on the left) show rough values.
Time	5, 30 sec 1, 3, 5, 10, 15, 30, 45, 60 min	Specifies the time from when a person leaves the monitor until the “No Presence detected” message appears. The monitor shifts to the power saving mode about 20 seconds after the message is displayed.
Sensitivity	Level 5	Regardless of whether the object is moving or not, the monitor cannot shift to the power saving mode if a person or a still object is present within the detection area. This setting is recommended if the monitor frequently shifts to the power saving, even though a user is in front of the screen.
	Level 4 Level 3 Level 2 Level 1	Since whether the object is moving or not is identified, the monitor can shift to the power saving mode even if there is a chair or wall in the detectable range. When the setting level is high (“Level 4” at maximum), even minor movements of a person are detected, causing the monitor to shift to the power saving mode only infrequently. On the other hand, when the setting level is low (“Level 1” at minimum), only major movements of a person are detected, allowing the monitor to shift to the power saving mode frequently. If the monitor does not shift to the power saving mode or recovers from the mode contrary to your expectations due to phenomenon such as the effect of lightening or ambient light, try decreasing the setting level.
Reset	-	All adjustments are reset to the default settings.

# Chapter 6 Troubleshooting

If a problem still remains after applying the suggested remedies, contact your local EIZO representative.

## 6-1. No Picture

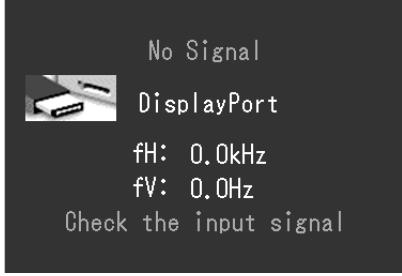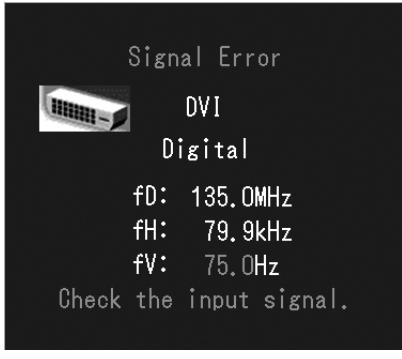Problem	Possible cause and remedy
<b>1. No picture</b> <ul style="list-style-type: none"><li>• Power indicator does not light.</li><li>• Power indicator is lighting orange and green.</li><li>• Power indicator is flashing orange and green.</li></ul>	<ul style="list-style-type: none"><li>• Check whether the power cord is connected properly.</li><li>• Turn the main power switch on.</li><li>• Press .</li><li>• Turn off the main power, and then turn it on again a few minutes later.</li><li>• Switch the input signal (see “<a href="#">Switching among input signals</a>” (page 35)).</li><li>• Move the mouse or press any key on the keyboard.</li><li>• Check whether the PC is turned on.</li><li>• When Presence Sensor is set to “Auto” or “Manual”, the monitor may have shifted to the power saving mode. Come near to the monitor.</li><li>• The device that is connected using DisplayPort have a problem. Solve the problem, turn off the monitor, and then turn it on again. Refer to the User’s Manual of the output device for further details.</li></ul>
<b>2. The message below appears.</b> <ul style="list-style-type: none"><li>• This message appears when no signal is input. Example: A dark gray rectangular screen with white text. At the top left is the word "No Signal". Below it is a small icon of a computer monitor. To its right is the word "DisplayPort". Underneath the icon are the text "fH: 0.0kHz" and "fV: 0.0Hz". At the bottom of the screen is the instruction "Check the input signal".</li><li>• The message shows that the input signal is out of the specified frequency range. Example: A dark gray rectangular screen with white text. At the top left is the word "Signal Error". Below it is a small icon of a computer monitor. To its right is the word "DVI". Underneath the icon is the word "Digital". Below these are the text "fD: 135.0MHz", "fH: 79.9kHz", and "fV: 75.0Hz". At the bottom of the screen is the instruction "Check the input signal.".</li></ul>	<p>This message appears when the signal is not input correctly even when the monitor functions properly.</p> <ul style="list-style-type: none"><li>• The message shown left may appear, because some PCs do not output the signal soon after power-on.</li><li>• Check whether the PC is turned on.</li><li>• Check whether the signal cable is connected properly.</li><li>• Switch the input signal (see “<a href="#">Switching among input signals</a>” (page 35)).</li></ul> <p>• Check whether the PC is configured to meet the resolution and vertical scan frequency requirements of the monitor (see “<a href="#">Compatible Resolutions</a>” in the Setup Manual.).</p> <ul style="list-style-type: none"><li>• Reboot the PC.</li><li>• Select an appropriate setting using the graphics board’s utility. Refer to the manual of the graphics board for details.</li></ul>

fD: Dot clock

fH: Horizontal scan frequency

fV: Vertical scan frequency

## 6-2. Imaging Problems

Problem	Possible cause and remedy
<b>1. The screen is too bright or too dark.</b>	<ul style="list-style-type: none"><li>Use "Brightness" in the Adjustment menu to adjust it. (The LCD monitor backlight has a limited life span. When the screen becomes dark or begins to flicker, contact your local EIZO representative.)</li></ul>
<b>2. Characters are blurred.</b>	<ul style="list-style-type: none"><li>Check whether the PC is configured to meet the resolution and vertical scan frequency requirements of the monitor (see "Compatible Resolutions" in the Setup Manual.).</li></ul>
<b>3. Afterimages appear.</b>	<ul style="list-style-type: none"><li>Afterimages are particular to LCD monitors. Avoid displaying the same image for a long time.</li><li>Use the screen saver or power saving function to avoid displaying the same image for extended periods of time.</li></ul>
<b>4. Interference patterns or pressure marks remain on the screen.</b>	<ul style="list-style-type: none"><li>Leave the monitor with a white or black screen. The symptom may disappear.</li></ul>

## 6-3. Other Problems

Problem	Possible cause and remedy
<b>1. The adjustment menu does not appear.</b>	<ul style="list-style-type: none"><li>Check whether the operation lock function is active (see "<a href="#">3-7. Locking Operation Buttons</a>" (page 21)).</li></ul>
<b>2. No mode menu is displayed.</b>	<ul style="list-style-type: none"><li>Check whether the operation lock function is active (see "<a href="#">3-7. Locking Operation Buttons</a>" (page 21)).</li></ul>
<b>3. (When using the presence sensor) The images remain to be displayed when you move away from the monitor. / The images do not reappear when you come near it.</b>	<ul style="list-style-type: none"><li>Check the setting environment of the monitor. Presence Sensor may not work correctly in the following environments.<ul style="list-style-type: none"><li>- There is an object that reflects light such as a mirror or glass in front of the sensor.</li><li>- The monitor is located in a place that receives direct sunlight.</li><li>- There is an infrared light emitting device near the monitor.</li><li>- There is some obstacle in front of the sensor.</li></ul></li><li>Check whether any dirt on the sensor. Clean the sensor with a soft cloth.</li><li>Check whether you are in front of the monitor. The infrared ray is emitted perpendicularly from the sensor to detect a person.</li></ul>
<b>4. The monitor connected with the USB cable is not detected. / USB devices connected to the monitor does not work.</b>	<ul style="list-style-type: none"><li>Check whether the USB cable is connected correctly (see "<a href="#">7-3. Making Use of USB (Universal Serial Bus)</a>" (page 37)).</li><li>Change the USB port to another one. If the PC or peripheral devices works correctly by changing the USB port, contact your local EIZO representative. (Refer to the manual of the PC for details.)</li><li>Reboot the PC.</li><li>If the peripheral devices work correctly when the PC and peripheral devices are connected directly, please contact your local EIZO representative.</li><li>Check whether the PC and its OS are USB compliant. (For USB compliance of the respective devices, consult their manufacturers.)</li><li>For Windows, check the PC's BIOS setting for USB. (Refer to the manual of the PC for details.)</li></ul>
<b>5. Integrated Front Sensor stays out.</b>	<ul style="list-style-type: none"><li>Turn the main power off and on again.</li></ul>
<b>6. SelfCalibration/Grayscale Check failure</b>	<ul style="list-style-type: none"><li>Refer to the error code table.</li><li>If an error code that does not appear in the error code table is displayed, contact your local EIZO representative.</li></ul>
<b>7. Time is not displayed correctly in the history of SelfCalibration/Grayscale Check</b>	<ul style="list-style-type: none"><li>Execute Monitor Detection using RadiCS/RadiCS LE. For details, refer to RadiCS/RadiCS LE User's Manual.</li></ul>

## ● Error code table

Error Code	Description
0***	<ul style="list-style-type: none"><li>• Errors that occurred during SelfCalibration</li></ul>
1***	<ul style="list-style-type: none"><li>• Errors that occurred during Grayscale Check</li></ul>
*1**	<ul style="list-style-type: none"><li>• Errors that occurred during DICOM</li></ul>
*2**	<ul style="list-style-type: none"><li>• Errors that occurred during CAL1</li></ul>
*3**	<ul style="list-style-type: none"><li>• Errors that occurred during CAL2</li></ul>
**10	<ul style="list-style-type: none"><li>• The product maximum brightness may be lower than the target brightness.</li><li>• Lower the target brightness.</li></ul>
**11	<ul style="list-style-type: none"><li>• The product minimum brightness may be higher than the target brightness.</li><li>• Raise the target brightness.</li></ul>
**34	<ul style="list-style-type: none"><li>• The sensor may not have come out during calibration, or light may have entered the sensor.</li><li>• Turn off the main power, wait for a few minutes before turning the power back on, and then execute SelfCalibration/Grayscale Check again.</li></ul>
**61	<ul style="list-style-type: none"><li>• The sensor may not have come out.</li><li>• Check whether there is any foreign object near the sensor.</li><li>• Execute SelfCalibration/Grayscale Check again.</li></ul>
**95	<ul style="list-style-type: none"><li>• The execution conditions for SelfCalibration/Grayscale Check may not be appropriate.</li><li>• Check the execution conditions and reset them if necessary.</li></ul>

# Chapter 7 Reference

## 7-1. Attaching Arm

An optional arm (or an optional stand) can be attached by removing the stand section. Please refer to our web site for the corresponding optional arm (or optional stand). <http://www.eizo.com>

### Attention

- When attaching an arm or stand, follow the instructions of their User's Manual.
- When using another manufacturer's arm or stand, confirm the following in advance and select one conforming to the VESA standard.
  - Clearance between the screw holes: 100 mm × 100 mm
  - Strong enough to support weight of the monitor unit (except the stand) and attachments such as cables.
- When using another manufacturer's arm or stand, use the following screws to secure it.
  - Screws fixing the stand to the monitor and screws supplied with this monitor
- When using an arm or stand, attach it to meet the following tilt angles of the monitor.
  - Up 45°, down 45°
- Connect the cables after attaching an arm or stand.
- Do not move the removed stand up and down. Doing so may result in injury or device damage.
- The monitor and arm or stand are heavy. Dropping them may result in injury or equipment damage.

### Procedure


**1. Lay the LCD monitor on a soft cloth spread over on a stable surface with the panel surface facing down.**

**2. Remove the stand.**

Prepare a screwdriver. Loosen the screws (four) securing the unit and the stand with the screwdriver.

**3. Attach the arm or stand to the monitor.**


Secure the monitor to the arm or stand using the screws specified in the User's Manual of the arm or stand.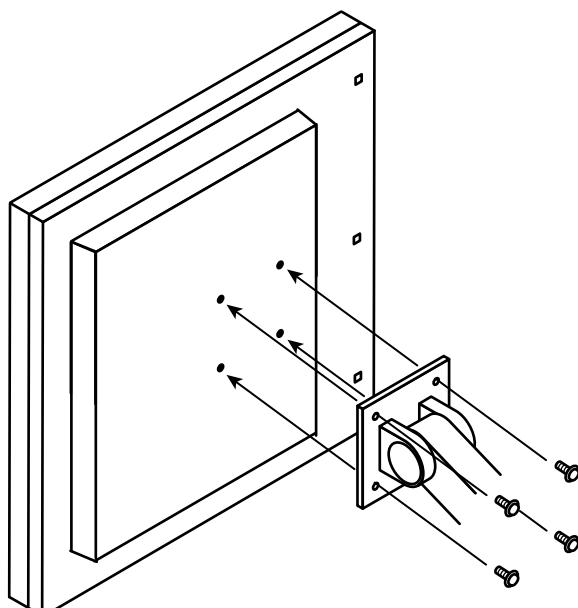

## 7-2. Connecting Multiple PCs

The product has multiple connections to PCs and allows you to switch to one of the connections for display.

### Connection examples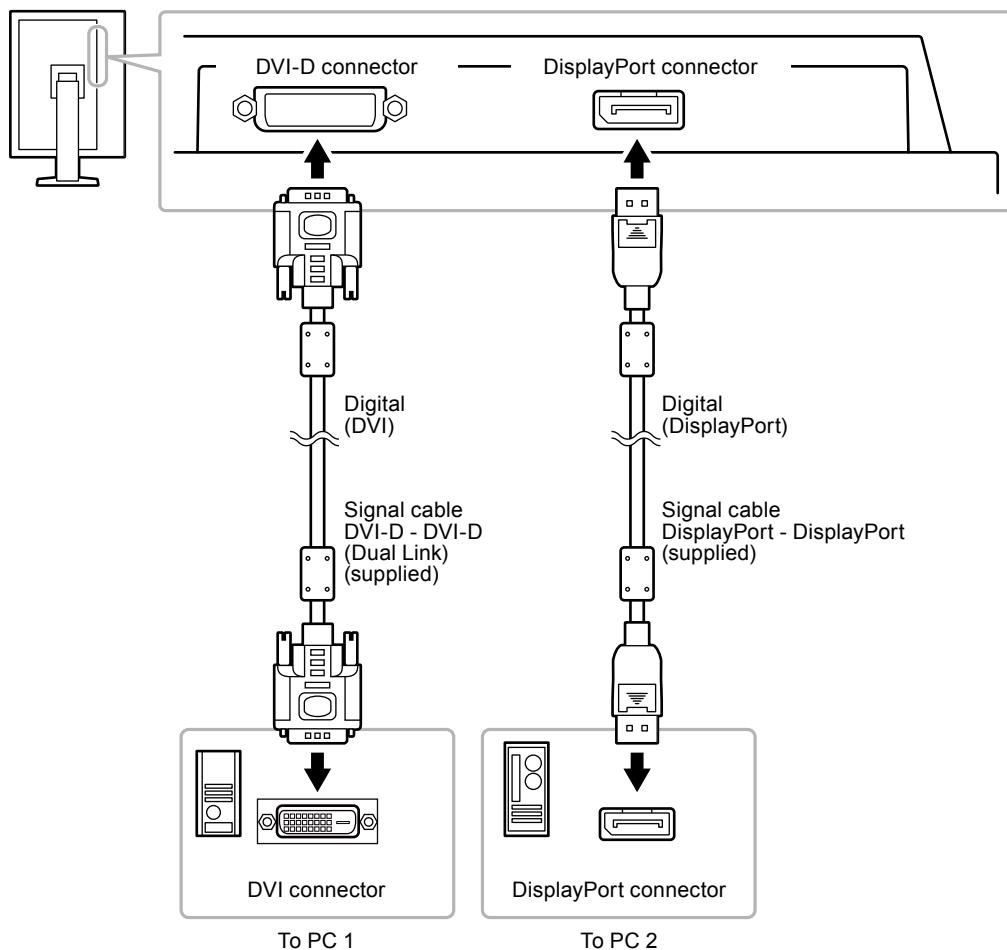


### ● Switching among input signals

#### Procedure

1. Select “Tools” from the Adjustment menu, and press ENTER.
2. Select “Input” from “Tools”, and press ENTER.
3. Select “DVI” or “DisplayPort” with ▲ or ▼.
4. Press ENTER to exit.

When the signal is switched, the active input port name appears at the top right corner of the screen.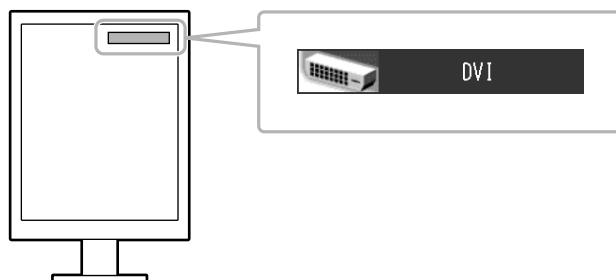

## ● Setting input signal switching procedure

Setting	Function
Auto	The monitor recognizes the connector through which PC signals are input. When a PC is turned off or enters the power saving mode, the monitor automatically displays another signal.
Manual	The monitor displays the signal from the selected connector despite the signal input. Select an active input signal with "Input".

### Procedure

1. Select "Tools" from the Adjustment menu, and press ENTER.
2. Select "Input Selection" from "Tools", and press ENTER.
3. Select "Auto" or "Manual" with ▲ or ▼.
4. Press ENTER to exit.

### Note

- If you select "Auto", the monitor enters the power saving mode after all the PCs have entered the power saving mode or have been switched off.

## 7-3. Making Use of USB (Universal Serial Bus)

This monitor has a USB hub. It works as a USB hub when connected to a USB-compatible PC, allowing the connection of peripheral USB devices.

### ● Required system environment


- A PC equipped with a USB port
- Windows 2000/XP/Vista/7/8 or Mac OS 9.2.2 and Mac OS X 10.2 or later
- USB cable

#### Attention

- This monitor may not work depending on PC, OS or peripheral devices to be used. For USB compatibility of peripheral devices, contact their manufacturers.
- Devices connected to the USB downstream port work when the monitor is in power saving mode or when the power button of the monitor is Off.
- When the main power of the monitor is Off, a device connected to the USB downstream port will not operate.

### ● Connection procedure (Setup of USB function)

1. Connect the monitor first to a PC using the signal cable, and run the PC.
2. Connect the USB cable between the USB downstream port of the PC and the USB upstream port of the monitor.  
The USB hub function is set up automatically upon connection of the USB cable.
3. Connect the peripheral USB devices to the USB downstream port of the monitor.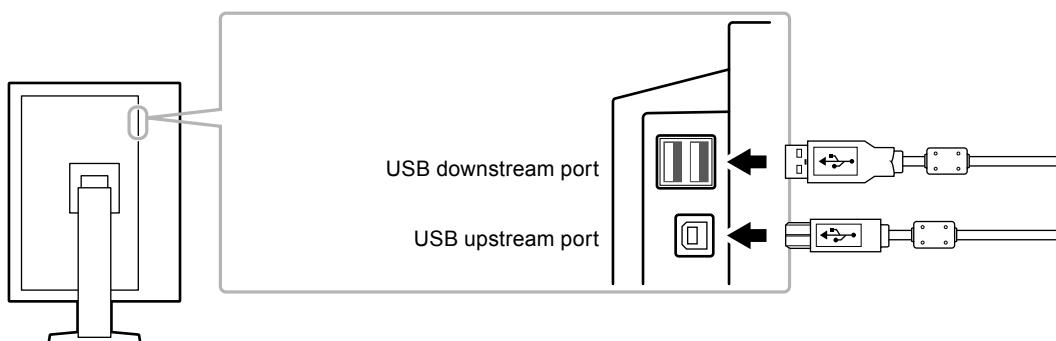

## 7-4. Showing Monitor Information

### ● Displaying signal information


The information about the current input signals can be displayed.

#### Procedure

1. Select “Tools” from the Adjustment menu, and press ENTER.
2. Select “Signal Info” from “Tools”, and press ENTER.

The “Signal Info” appears.

(Example)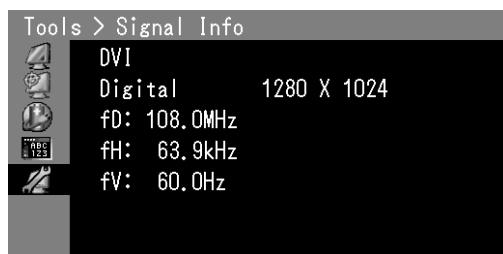

### ● Showing monitor information

The information about the monitor can be displayed.


#### Procedure

1. Select “Tools” from the Adjustment menu, and press ENTER.
2. Select “Monitor Info” from “Tools”, and press ENTER.

The “Monitor Info” appears.

The model name, serial number, firmware version, and usage time can be displayed.

(Example)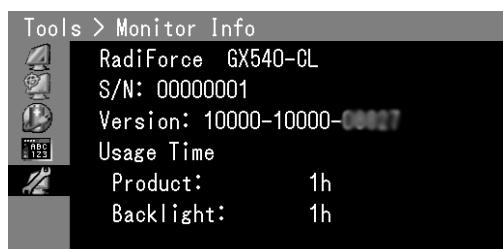

#### Attention

- When you turn on the monitor for the first time, the usage time may be more than 0 due to a factory inspection.

## 7-5. Specifications

LCD Panel	Size	21.3-inch (540 mm)	
	Type	TFT monochrome LCD, LED backlight	
	Surface treatment	Anti-glare	
	Surface hardness	2H	
	Viewing angles	Horizontal 176°, vertical 176° (CR≥10)	
	Dot pitch	0.165 mm	
	Response time	Black-white-black: Approx. 25 ms	
Horizontal scan frequency		31-135kHz	
Vertical scan frequency		DVI:	24 - 61 Hz (non-interlace) (VGA TEXT: 69 to 71 Hz, QSXGA (2048 × 2560): 24 to 51 Hz)
		DP:	49 - 61 Hz (non-interlace) (VGA TEXT: 69 to 71 Hz, QSXGA (2048 × 2560): 49 to 51 Hz)
Resolution		5M pixels (Portrait orientation: 2048 dots × 2560 lines (H × V))	
Max. dot clock		290MHz	
Display grayscales		1,024 steps of 16,369 steps	
Recommended Brightness		500 cd/m <sup>2</sup>	
Display area (H × V)		337.9 mm (13.30 inch) × 422.4 mm (16.63 inch) (Portrait orientation)	
Power supply		100 - 120 Vac ±10%, 50/60 Hz 1.1 - 0.9A 200 - 240 Vac ±10%, 50/60 Hz 0.6 - 0.5A	
Power consumption	Screen display On	108 W or less	
	Power saving mode	0.7 W or less	(when only the DVI signal connector is connected, no USB device is connected, "Input Selection" is set to "Manual", "DC5V Output" is set to "Off", and "DP Power Save" is set to "On")
	Power Off	0.5 W or less	(when no USB device is connected, "DC5V Output" is set to "Off", and "DP Power Save" is set to "On")
	Main power Off	0 W	
Input signal connectors		DVI-D connector	
		DisplayPort connector (Standard V1.1a, applicable to HDCP)	
Digital Signal (DVI) Transmission System		TMDS (Single Link / Dual Link)	
Plug & Play		Digital (DVI-D): VESA DDC 2B / EDID structure 1.3 Digital (DisplayPort) : VESA DisplayPort / EDID structure 1.4	
Dimensions (width) × (height) × (depth)	Main unit	388 mm (15.3 inch) × 512 - 595 mm (20.2 - 23.4 inch) × 245.5 mm (9.7 inch)	
	Main unit (without stand)	388 mm (15.3 inch) × 496 mm (19.5 inch) × 99 mm (3.9 inch)	
Mass	Main unit	Approx. 11.5 kg (25.4 lbs.)	
	Main unit (without stand)	Approx. 8.8 kg (19.4 lbs.)	
Movable range	FlexStand	Tilt:	Up 30°, down 0°
		Swivel:	344°
		Adjustable height:	174 mm (Tilt: 30°), 83 mm (Tilt: 0°)
		Rotation:	90° (counterclockwise for portrait display)
Environmental conditions	Temperature	Operating:	0°C to 35°C (32°F to 95°F)
		Transportation/Storage:	-20°C to 60°C (-4°F to 140°F)
	Humidity	Operating:	20% to 80% R.H. (no condensation)
		Transportation/Storage:	10% to 90% R.H. (no condensation)
Air pressure	Operating:	700 hPa to 1060 hPa	
	Transportation/Storage:	200 hPa to 1060 hPa	

USB	Standard	USB Specification Revision 2.0
	Port	Upstream port × 1, Downstream port × 2
	Communication Speed	480 Mbps (high), 12 Mbps (full) 1.5 Mbps (low)
	Supply current	Downstream: Max. 500 mA/1 port

## Main Default Settings

**CAL Switch Mode: The default display setting is 1-DICOM.**

Mode	Brightness	Gamma value
DICOM	500 cd/m <sup>2</sup>	DICOM
CAL1	400 cd/m <sup>2</sup>	DICOM
CAL2	300 cd/m <sup>2</sup>	DICOM

## Others

Input <sup>*1</sup>	DVI
Presence Sensor	Off
Power Save	High
DP Power Save <sup>*1</sup>	Off
Input Selection	Auto
Logo <sup>*1</sup>	Off
Key Lock <sup>*1</sup>	Off
DDC <sup>*1</sup>	On
DC5V Output <sup>*1</sup>	Off
Orientation <sup>*1</sup>	Portrait(HW)
Signal Format <sup>*1</sup>	Standard
Sub Pixel Drive <sup>*1,*2</sup>	Off
Menu Position	Center
Language	English

\*1 These menus cannot be initialized with “All Reset” (see [page 23](#)).


\*2 Leave the default setting “Off” as it is.

### Attention

- If “Sub Pixel Drive” is set to “On”, the screen is not displayed properly.

## Outside Dimensions

Unit: mm (inch)

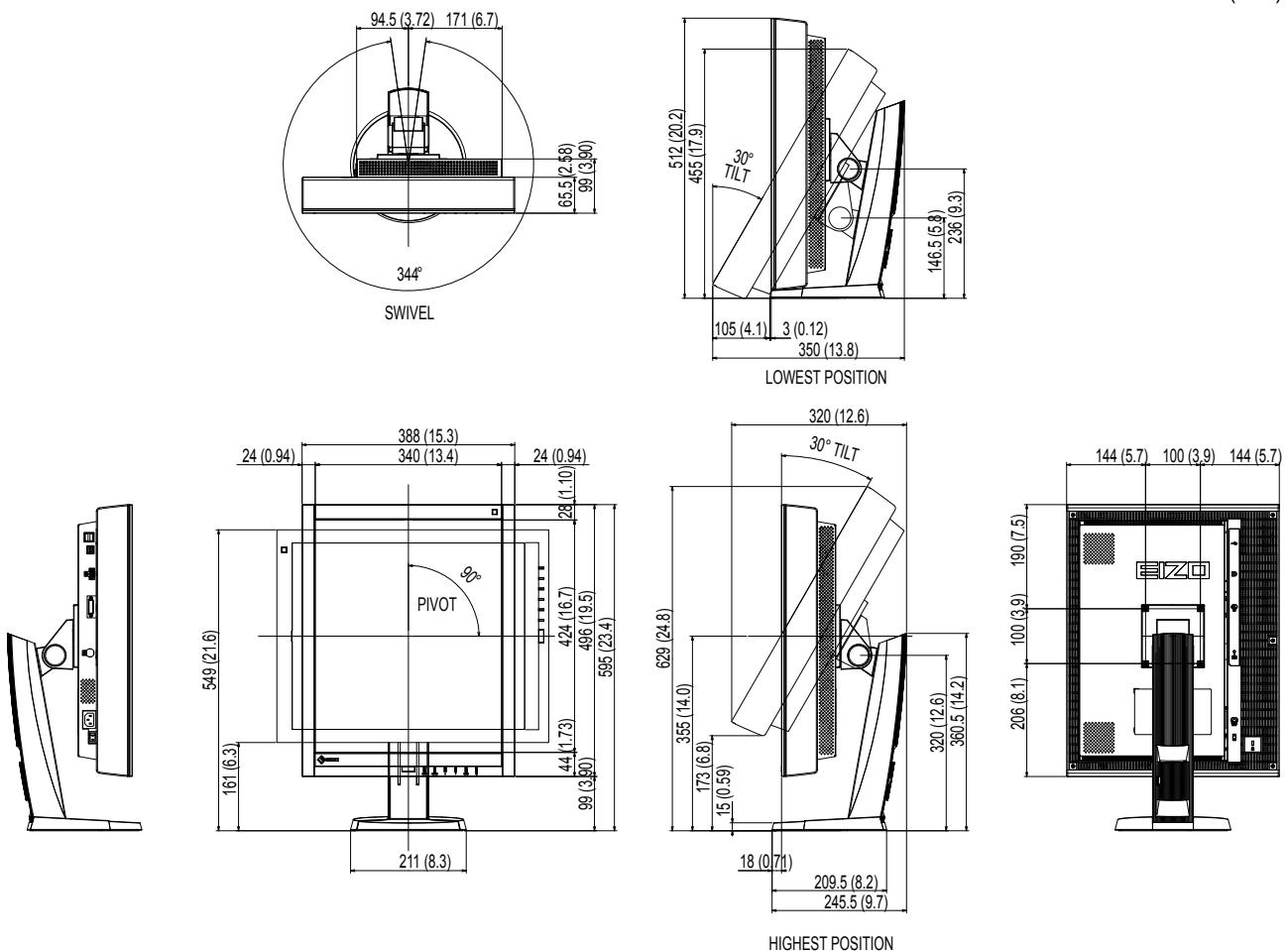## Accessories

Calibration Kit	EIZO "RadiCS UX1" Ver. 4.1.4 or later EIZO "RadiCS Version Up Kit" Ver. 4.1.4 or later
Network QC Management Software	EIZO "RadiNET Pro" Ver. 4.1.4 or later
Cleaning Kit	EIZO "ScreenCleaner"

For the latest information about the accessories and information about the latest compatible graphics board, refer to our web site.

<http://www.eizo.com>

# Chapter 8 Glossary

## DDC (Display Data Channel)

VESA provides the standardization for the interactive communication of the setting information, etc. between a PC and the monitor.

## DICOM (Digital Imaging and Communication in Medicine)

The DICOM standard was developed by the American College of Radiology and the National Electrical Manufacturer's Association of the USA.

The DICOM compatible device connection enables to transfer the medical image and information. The DICOM, Part 14 document defines the digital, grayscale medical image display.

## DisplayPort

DisplayPort is a next-generation digital AV interface that allows connection of the PC, audio, imaging devices, etc. to the monitor. One cable can transfer sound with images.

## DVI (Digital Visual Interface)

DVI is a digital interface standard. DVI allows direct transmission of the PC's digital data without loss. This adopts the TMDS transmission system and DVI connectors. There are two types of DVI connectors. One is a DVI-D connector for digital signal input only. The other is a DVI-I connector for both digital and analog signal inputs.

## DVI DMPM (DVI Digital Monitor Power Management)

DVI DMPM is a digital interface power saving function. The "Monitor ON (operating mode)" and "Active Off (power saving mode)" are indispensable for DVI DMPM as the monitor's power mode.

## HDCP (High-bandwidth Digital Content Protection)

Digital signal coding system developed to copy-protect the digital contents, such as video, music, etc. This helps to transmit the digital contents safely by coding the digital contents sent via the DVI or HDMI connector on the output side and decoding them on the input side. Any digital contents cannot be reproduced if both of the equipments on the output and input sides are not applicable to HDCP system.

## Resolution

The LCD panel consists of numerous pixels of specified size, which are illuminated to form images. This monitor consists of horizontal 2048 pixels and 2560 vertical pixels. At a resolution of 2048 × 2560 (Portrait) and 2560 × 2048 (landscape), all pixels are illuminated as a full screen (1:1).

## TMDS (Transition Minimized Differential Signaling)

A signal transmission system for digital interface.

# Appendix

## Trademark

The terms HDMI and HDMI High-Definition Multimedia Interface, and the HDMI Logo are trademarks or registered trademarks of HDMI Licensing, LLC in the United States and other countries.

The DisplayPort Compliance Logo and VESA are registered trademarks of the Video Electronics Standards Association.

Acrobat, Adobe, Adobe AIR, and Photoshop are registered trademarks of Adobe Systems Incorporated in the United States and other countries.

AMD Athlon and AMD Opteron are trademarks of Advanced Micro Devices, Inc.

Apple, ColorSync, eMac, iBook, iMac, iPad, Mac, MacBook, Macintosh, Mac OS, PowerBook, and QuickTime are registered trademarks of Apple Inc.

ColorMunki, Eye-One, and X-Rite are registered trademarks or trademarks of X-Rite Incorporated in the United States and/or other countries.

ColorVision and ColorVision Spyder2 are registered trademarks of DataColor Holding AG in the United States. Spyder3 and Spyder4 are trademarks of DataColor Holding AG.

ENERGY STAR is a registered trademark of the United States Environmental Protection Agency in the United States and other countries.

GRACoL and IDEAlliance are registered trademarks of International Digital Enterprise Alliance.

NEC is a registered trademark of NEC Corporation.

PC-9801 and PC-9821 are trademarks of NEC Corporation.

NextWindow is a registered trademark of NextWindow Ltd.

Pentium is a registered trademark of Intel Corporation in the United States and other countries.

PowerPC is a registered trademark of International Business Machines Corporation.

PlayStation is a registered trademark of Sony Computer Entertainment Inc.

PSP and PS3 are trademarks of Sony Computer Entertainment Inc.

RealPlayer is a registered trademark of RealNetworks, Inc.

TouchWare is a trademark of 3M Touch Systems, Inc.

Windows, Windows Media, Windows Vista, SQL Server, and Xbox 360 are registered trademarks of Microsoft Corporation in the United States and other countries.

Kensington and MicroSaver are trademarks of ACCO Brands Corporation.

EIZO, the EIZO Logo, ColorEdge, DuraVision, FlexScan, FORIS, RadiCS, RadiForce, RadiNET, Raptor, and ScreenManager are registered trademarks of EIZO Corporation in Japan and other countries.

ColorNavigator, EcoView NET, EIZO EasyPIX, EIZO ScreenSlicer, i•Sound, Screen Administrator, and UniColor Pro are trademarks of EIZO Corporation.

All other company and product names are trademarks or registered trademarks of their respective owners.

## License

A round gothic bold bit map font designed by Ricoh is used for the characters displayed on this product.

# **Medical Standard**

---

- It shall be assured that the final system is in compliance to IEC60601-1-1 requirement.
- Power supplied equipment can emit electromagnetic waves, that could influence, limit or result in malfunction of the monitor. Install the equipment in a controlled environment, where such effects are avoided.

## **Classification of Equipment**

- Type of protection against electric shock : Class I
- EMC class: EN60601-1-2:2007 Group 1 Class B
- Classification of medical device (MDD 93/42/EEC) : Class I
- Mode of operation : Continuous
- IP Class : IPX0

# FCC Declaration of Conformity

For U.S.A. , Canada, etc. (rated 100-120 Vac) Only

## FCC Declaration of Conformity

We, the Responsible Party

EIZO Inc.

5710 Warland Drive, Cypress, CA 90630

Phone: (562) 431-5011

declare that the product

Trade name: EIZO

Model: RadiForce GX540

is in conformity with Part 15 of the FCC Rules. Operation of this product is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures.

- \* Reorient or relocate the receiving antenna.
- \* Increase the separation between the equipment and receiver.
- \* Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- \* Consult the dealer or an experienced radio/TV technician for help.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

### Note

Use the attached specified cable below or EIZO signal cable with this monitor so as to keep interference within the limits of a Class B digital device.

- AC Cord
- Shielded Signal Cable (enclosed)

## Canadian Notice

This Class B digital apparatus complies with Canadian ICES-003.

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# EMC Information

Essential performance of RadiForce series is to display images and operate functions normally.

## CAUTION

The RadiForce series requires special precautions regarding EMC and need to be installed, put into service and used according to the following information.

**Do not use any cables other than the cables that provided or specified by us.**

**Using other cables may cause the increase of emission or decrease of immunity.**

**Do not put any portable and mobile RF communications equipment close to the RadiForce series. Doing so may affect the RadiForce series.**

**The RadiForce series should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the equipment or system should be observed to verify normal operation in the configuration in which it will be used.**

**Anyone who connects additional equipment to the signal input part or signal output parts, configuring a medical system, is responsible that the system complies with the requirements of IEC/EN60601-1-2.**

### Guidance and manufacturer's declaration - electromagnetic emissions

The RadiForce series is intended for use in the electromagnetic environment specified below.

The customer or the user of the RadiForce series should assure that it is used in such an environment.

Emission test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR11/EN55011	Group 1	The RadiForce series uses RF energy only for its internal function. Therefore, its RF emission are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR11/EN55011	Class B	The RadiForce series is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC/EN61000-3-2	Class D	
Voltage fluctuations / flicker emissions IEC/EN61000-3-3	Complies	

### Guidance and manufacturer's declaration - electromagnetic immunity

The RadiForce series is intended for use in the electromagnetic environment specified below.

The customer or the user of the RadiForce series should assure that it is used in such an environment.

Immunity test	IEC/EN60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC/EN61000-4-2	±6kV contact ±8kV air	±6kV contact ±8kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient / burst IEC/EN61000-4-4	±2kV for power supply lines ±1kV for input/output lines	±2kV for power supply lines ±1kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC/EN61000-4-5	±1kV line(s) to line(s) ±2kV line(s) to earth	±1kV line(s) to line(s) ±2kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC/EN61000-4-11	<5% $U_T$ (>95% dip in $U_T$ ) for 0.5 cycle 40% $U_T$ (60% dip in $U_T$ ) for 5 cycles 70% $U_T$ (30% dip in $U_T$ ) for 25 cycles <5% $U_T$ (>95% dip in $U_T$ ) for 5sec	<5% $U_T$ (>95% dip in $U_T$ ) for 0.5 cycle 40% $U_T$ (60% dip in $U_T$ ) for 5 cycles 70% $U_T$ (30% dip in $U_T$ ) for 25 cycles <5% $U_T$ (>95% dip in $U_T$ ) for 5sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the RadiForce series requires continued operation during power mains interruptions, it is recommended that the RadiForce series be powered from an uninterruptible power supply or a battery.
Power frequency (50/60Hz) magnetic field IEC/EN61000-4-8	3A/m	3A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE  $U_T$  is the a.c. mains voltage prior to application of the test level.

### Guidance and manufacturer's declaration - electromagnetic immunity

The RadiForce series is intended for use in the electromagnetic environment specified below. The customer or the user of the RadiForce series should assure that it is used in such an environment.

Immunity test	IEC/EN60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC/EN61000-4-6	3Vrms 150kHz to 80MHz	3Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the RadiForce series, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.  Recommended Separation distance  $d = 1.2 \sqrt{P}$  $d = 1.2 \sqrt{P}, 80MHz to 800MHz$ $d = 2.3 \sqrt{P}, 800MHz to 2.5GHz$  Where "P" is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and "d" is the recommended separation distance in meters (m).  Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey <sup>a</sup> , should be less than the compliance level in each frequency range <sup>b</sup> .  Interference may occur in the vicinity of equipment marked with the following symbol. 
			NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.  NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

<sup>a</sup> Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the RadiForce series is used exceeds the applicable RF compliance level above, the RadiForce series should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the RadiForce series.
<sup>b</sup> Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

### **Recommended separation distances between portable and mobile RF communications equipment and the RadiForce Series**

The RadiForce series is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the RadiForce series can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the RadiForce series as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150kHz to 80MHz $d = 1.2 \sqrt{P}$	80MHz to 800MHz $d = 1.2 \sqrt{P}$	800MHz to 2.5GHz $d = 2.3 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance “d” in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where “P” is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

<b>Cable length</b>	
Signal Cable (DD300DL) :	3.0m
Signal Cable (PP300) :	3.0m
USB Cable (UU300) :	3.0m

# LIMITED WARRANTY

---

## LIMITED WARRANTY

EIZO Corporation (hereinafter referred to as "**EIZO**") and distributors authorized by EIZO (hereinafter referred to as the "**Distributors**") warrant, subject to and in accordance with the terms of this limited warranty (hereinafter referred to as the "**Warranty**"), to the original purchaser (hereinafter referred to as the "**Original Purchaser**") who purchased the product specified in this document (hereinafter referred to as the "**Product**") from EIZO or Distributors, that EIZO and Distributors shall, at their sole discretion, either repair or replace the Product at no charge if the Original Purchaser becomes aware within the Warranty Period (defined below) that (i) the Product malfunctions or is damaged in the course of normal use of the Product in accordance with the description in the instruction manual attached to the Product (hereinafter referred to as the "**User's Manual**"), or (ii) the LCD panel and brightness of the Product cannot maintain the recommended brightness specified in the User's Manual in the course of normal use of the Product in accordance with the description of the User's Manual.

The period of this Warranty is five (5) years from the date of purchase of the Product (hereinafter referred to as the "Warranty Period"), provided that the brightness of the Product shall be warranted only if the Product has been used within the recommended brightness described in the User's Manual and the Warranty Period for the brightness is limited to five (5) years from the date of purchase of the Product. EIZO and Distributors shall bear no liability or obligation with regard to the Product in relation to the Original Purchaser or any third parties other than as provided under this Warranty.

EIZO and Distributors will cease to hold or store any parts of the Product upon expiration of seven (7) years after the production of such parts is discontinued. In repairing the monitor, EIZO and Distributors will use renewal parts which comply with our QC standards.

The Warranty is valid only in the countries or territories where the Distributors are located. The Warranty does not restrict any legal rights of the Original Purchaser.

Notwithstanding any other provision of this Warranty, EIZO and Distributors shall have no obligation under this Warranty whatsoever in any of the cases as set forth below:

- (a) Any defect of the Product caused by freight damage, modification, alteration, abuse, misuse, accident, incorrect installation, disaster, faulty maintenance and/or improper repair by third party other than EIZO and Distributors;
- (b) Any incompatibility of the Product due to possible technical innovations and/or regulations;
- (c) Any deterioration of the sensor;
- (d) Any deterioration of display performance caused by the deterioration of expendable parts such as the LCD panel and/or backlight, etc. (e.g. changes in color, changes in color uniformity, defects in pixels including burnt pixels, etc.);
- (e) Any deterioration of the Product caused by the use at higher brightness than the recommended brightness described in the User's Manual;
- (f) Any defect of the Product caused by external equipment;
- (g) Any defect of the Product on which the original serial number has been altered or removed;
- (h) Any normal deterioration of the product, particularly that of consumables, accessories, and/or attachments (e.g. buttons, rotating parts, cables, User's Manual, etc.); and
- (i) Any deformation, discoloration, and/or warp of the exterior of the product including that of the surface of the LCD panel.

To obtain service under the Warranty, the Original Purchaser must deliver the Product, freight prepaid, in its original package or other adequate package affording an equal degree of protection, assuming the risk of damage and/or loss in transit, to the local Distributor. The Original Purchaser must present proof of purchase of the Product and the date of such purchase when requesting services under the Warranty.

The Warranty Period for any replaced and/or repaired product under this Warranty shall expire at the end of the original Warranty Period.

**EIZO OR DISTRIBUTORS ARE NOT RESPONSIBLE FOR ANY DAMAGE TO, OR LOSS OF, DATA OR OTHER INFORMATION STORED IN ANY MEDIA OR ANY PART OF ANY PRODUCT RETURNED TO EIZO OR DISTRIBUTORS FOR REPAIR.**

EIZO AND DISTRIBUTORS MAKE NO FURTHER WARRANTIES, EXPRESSED OR IMPLIED, WITH RESPECT TO THE PRODUCT AND ITS QUALITY, PERFORMANCE, MERCHANTABILITY OR FITNESS FOR ANY PARTICULAR USE. IN NO EVENT SHALL EIZO OR DISTRIBUTORS BE LIABLE FOR ANY INCIDENTAL, INDIRECT, SPECIAL, CONSEQUENTIAL OR OTHER DAMAGE WHATSOEVER (INCLUDING, WITHOUT LIMITATION, DAMAGES FOR LOSS OF PROFIT, BUSINESS INTERRUPTION, LOSS OF BUSINESS INFORMATION, OR ANY OTHER PECUNIARY LOSS) ARISING OUT OF THE USE OR INABILITY TO USE THE PRODUCT OR IN ANY CONNECTION WITH THE PRODUCT, WHETHER BASED ON CONTRACT, TORT, NEGLIGENCE, STRICT LIABILITY OR OTHERWISE, EVEN IF EIZO OR DISTRIBUTORS HAVE BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. THIS EXCLUSION ALSO INCLUDES ANY LIABILITY WHICH MAY ARISE OUT OF THIRD PARTY CLAIMS AGAINST THE ORIGINAL PURCHASER. THE ESSENCE OF THIS PROVISION IS TO LIMIT THE POTENTIAL LIABILITY OF EIZO AND DISTRIBUTORS ARISING OUT OF THIS LIMITED WARRANTY AND/OR SALES.

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---

## **Recycling Information**

This product, when disposed of, is supposed to be collected and recycled according to your country's legislation to reduce environmental burden. When you dispose of this product, please contact a distributor or an affiliate in your country.

The contact addressees are listed on the EIZO website below.

<http://www.eizo.com>

For recycling information for customers in Switzerland, please refer to the following website.

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