Instructions for Use

RadiLight™

Comfort Light for reading rooms

Please read this "Instructions for Use" carefully to familiarize yourself with safe and effective usage.
Please retain this manual for future reference.

SAFETY SYMBOLS

This manual and this product use the safety symbols below. They denote critical information. Please read them carefully.

 WARNING Failure to abide by the information in a WARNING may result in serious injury and can be life threatening.	 CAUTION Failure to abide by the information in a CAUTION may result in moderate injury and/or property or product damage.
	Indicates a warning or caution.
	Indicates a prohibited action. For example,  means "Do not disassemble".

PRECAUTIONS

WARNING

If the unit begins to emit smoke, smells like something is burning, or makes strange noises, disconnect all power connections immediately and contact your EIZO representative for advice.
Attempting to use a malfunctioning unit may result in fire, electric shock, or equipment damage.

Do not open the cabinet or modify the unit.

Opening the cabinet or modifying the unit may result in fire, electric shock, or burn.

Refer all servicing to qualified service personnel.

Do not attempt to service this product yourself as opening or removing covers may result in fire, electric shock, or equipment damage.

Use the unit in an appropriate location.

Otherwise, fire, electric shock, or equipment damage may result.

- Do not place outdoors.
- Do not place in any form of transportation (ships, aircraft, trains, automobiles, etc.).
- Do not place in dusty or humid environments.
- Do not place in locations where water may be splashed on the screen (bathrooms, kitchens, etc.).
- Do not place in locations where steam comes in direct contact with the screen.
- Do not place near heat generating devices or humidifiers.
- Do not place in locations where the product is subject to direct sunlight.
- Do not place in environments with inflammable gas.
- Do not place in environments with corrosive gases (such as sulfur dioxide, hydrogen sulfide, nitrogen dioxide, chlorine, ammonia, and ozone).
- Do not place in environments with dust, components that accelerate corrosion in the atmosphere (such as sodium chloride and sulfur), conductive metals, and so on.

CAUTION

Do not look directly into the light source.

Doing so may cause eye pain or impair vision.

Do not apply excessive force to the arm.

If you bend or twist the arm using excessive force, it may break or malfunction.

When rotating the monitor, remove the lighting unit.

If the lighting unit is not removed, it may come off when rotating the monitor, or the arm may accidentally hit other components such as the monitor and cause components to break or malfunction.

Dispose of the products in accordance with the laws and regulations of your region.

Symbol on the unit

	Direct current
---	----------------

No part of this manual may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, or otherwise, without the prior written permission of EIZO Corporation. EIZO Corporation is under no obligation to hold any submitted material or information confidential unless prior arrangements are made pursuant to EIZO Corporation's receipt of said information. Although every effort has been made to ensure that this manual provides up-to-date information, please note that EIZO product specifications are subject to change without notice.


EIZO, the EIZO Logo and RadiForce are registered trademarks of EIZO Corporation in Japan and other countries.

RadiLight is a trademark of EIZO Corporation.


All other company names, product names, and logos are trademarks or registered trademarks of their respective owners.

Checking the Contents of the Package


Confirm that the following items are included in the package. If any items are missing or damaged, contact your local EIZO representative.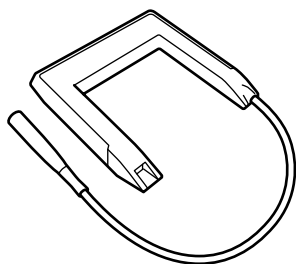


Lighting unit (main unit)

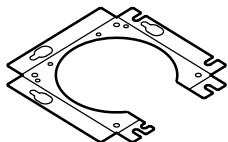
Mounting bracket

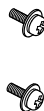Bracket mounting bolts
(M3x8) x2

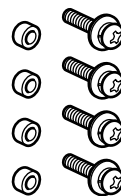Spacers x4
Bracket mounting bolts
(M4x18) x4

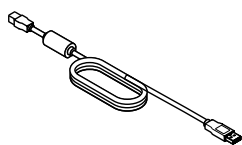Dedicated cable

User's Manual

Compatible Monitors

This product is compatible with EIZO RadiForce series monitors.


For detailed information about supported monitors, refer to the EIZO website. www.eizoglobal.com

Installation and Connection Procedures


Mounting Holes for RadiLight

The installation process differs depending on whether there are mounting holes to install RadiLight. Check your monitor for the mounting holes.

Monitors with mounting holes for RadiLight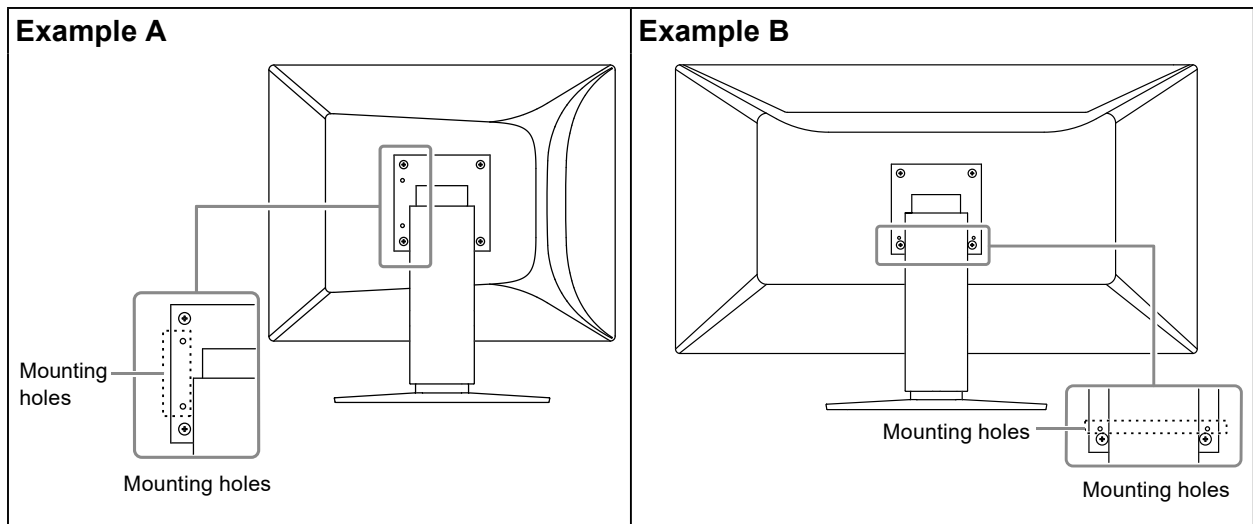

Monitors without mounting holes for RadiLight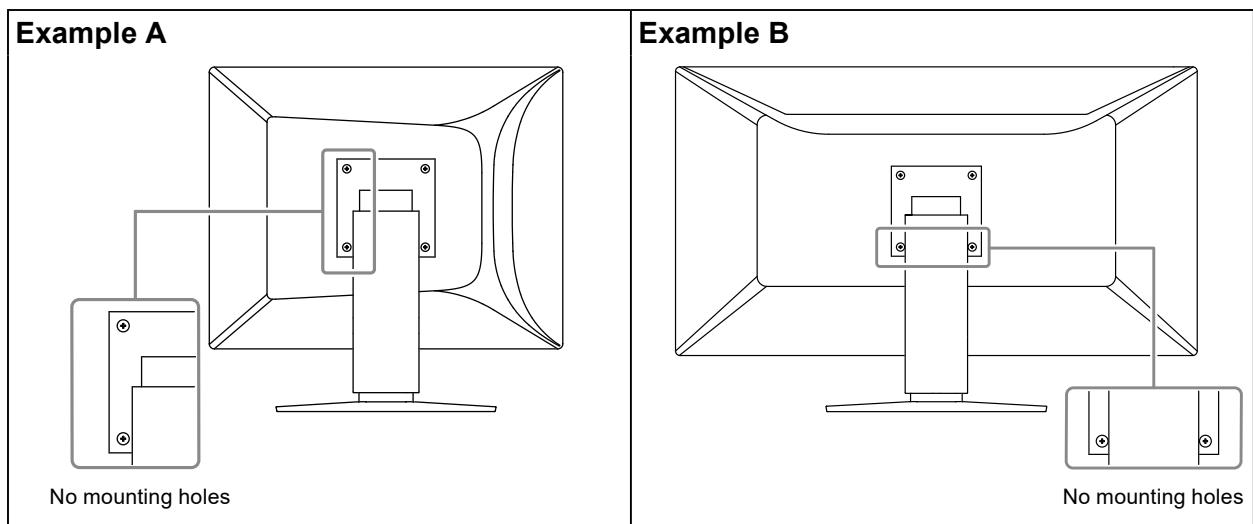


Attaching the Mounting Bracket to the LCD Monitor

Ensure that the monitor is powered off.

● Monitors with mounting holes for RadiLight

1. Align the holes of the mounting bracket with the holes of the monitor and stand. Fasten the bracket using the M3x8 screws (two locations).


* Prepare a screwdriver.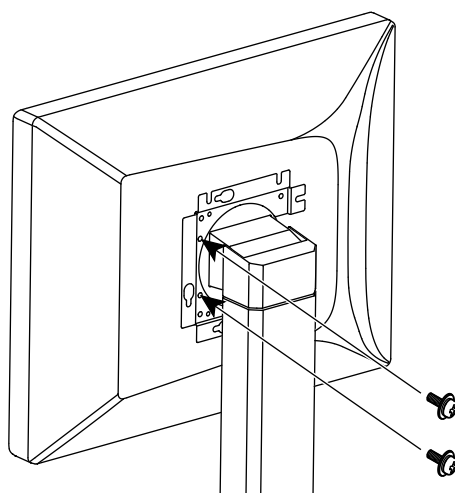

● Monitors without mounting holes for RadiLight

1. Lay the LCD monitor on a soft cloth spread over a stable surface with the panel surface facing down.
2. Unscrew the screws (four) securing the monitor and the stand.

* Prepare a screwdriver.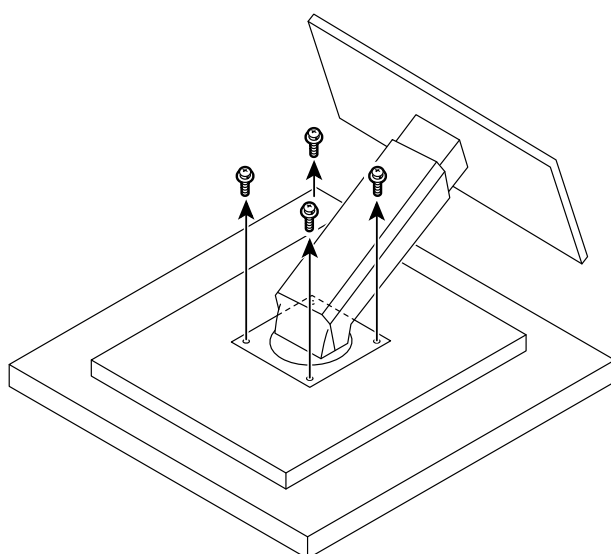

3. Attach the mounting bracket.

For a monitor with an indented area of a size that fits the attachment part of the stand

1. Place the spacers over the screw holes (four locations) in the stand. (Figure 1)
2. Insert the bracket from the side of the stand, and turn the mounting bracket so that its open side faces the top of the monitor. (Figure 2)
3. Align the screw holes in the monitor, mounting bracket, and stand, and fix them using the supplied screws (M4x18) (four locations). (Figure 3)

Figure 1

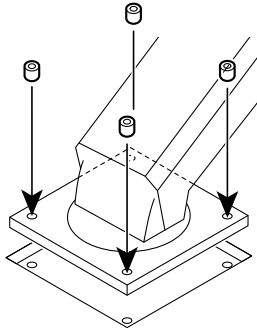
Figure 2


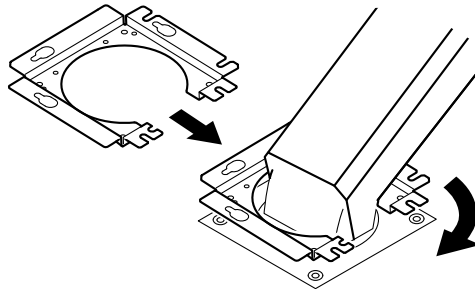
Figure 3

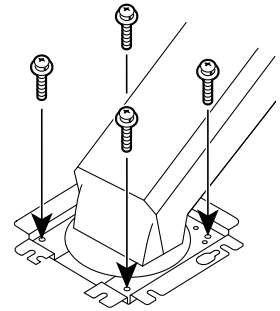For a monitor on which the part to attach the stand to is flat or bulging

1. Place the mounting bracket so that its open side faces the top of the monitor. (Figure 1)
2. Align the screw holes in the monitor, mounting bracket, and stand, and fix them using the screws that have been removed (four locations). (Figure 2)

Figure 1


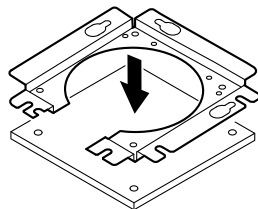
Figure 2

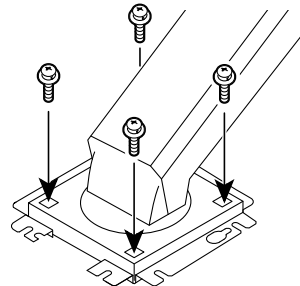
Attaching the Lighting Unit

1. Stand the monitor upright.

2. Attach the lighting unit to the mounting bracket.

Hook the protruding catches on the lighting unit to the holes in the mounting bracket (four locations). Slide the lighting unit down so that the catches securely position the lighting unit.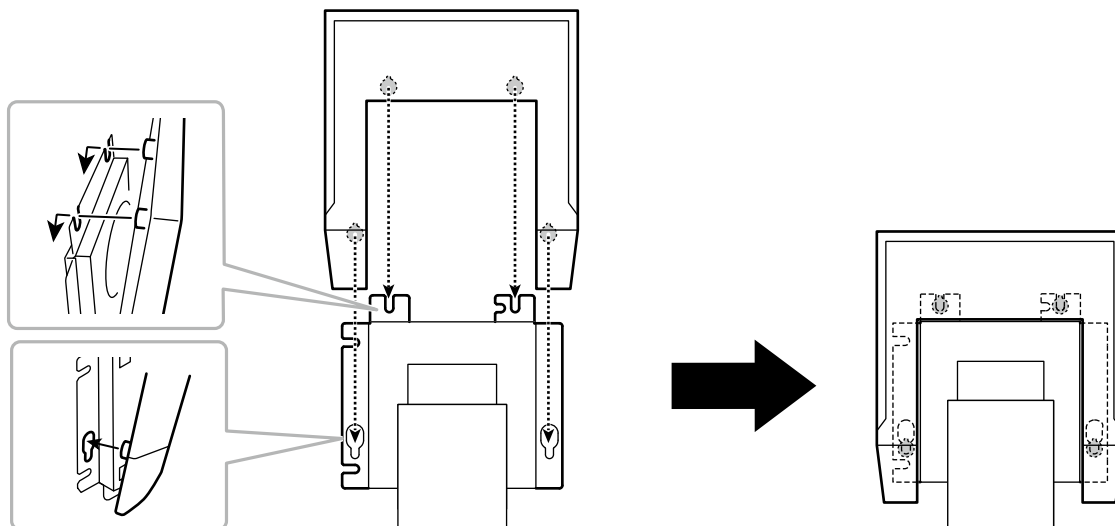

Note

- To remove the lighting unit, slide it straight up.


3. Connect the cable as follows.

RadiLight and monitor

Connecting cable: Dedicated cable

Monitor and PC

Connecting cable: USB cable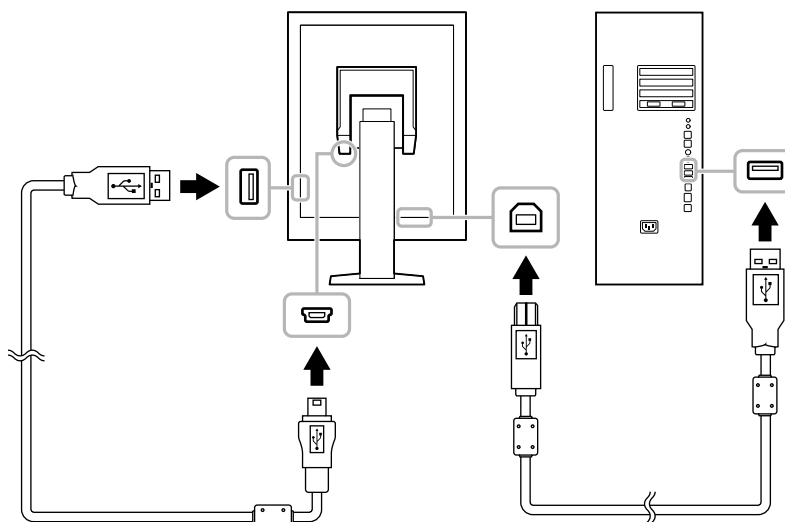


Attention

- If the cable is connected to a port other than the monitor USB port such as an AC adapter or PC USB port, you may encounter the following problematic symptoms.
Ensure that you connect the cable to the monitor USB port.
 - Does not power on.
 - Malfunction occurs.
- Always use the dedicated cable to connect to the monitor.

Functions and Operations


Name and functions of each part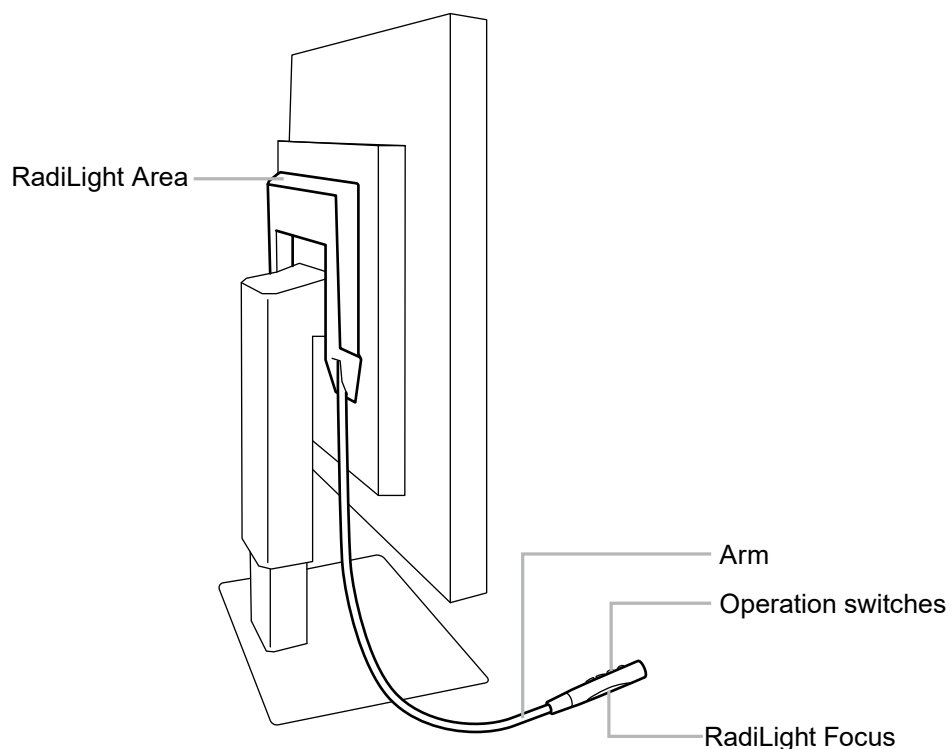

- RadiLight Area: Indirectly lights up the room from behind the monitor.
- RadiLight Focus: Lights up the area around your hands and desk top.
- Arm: Adjusts the position of the operation switches and RadiLight Focus.
- Operation switches: Functions as follows.
 - Turns RadiLight Area and RadiLight Focus on/off
 - Adjusts the brightness of RadiLight Area

Operating steps

1. Turn on the monitor and PC.
2. The operation switches turn RadiLight Area and RadiLight Focus on/off. Adjusts the brightness of RadiLight Area.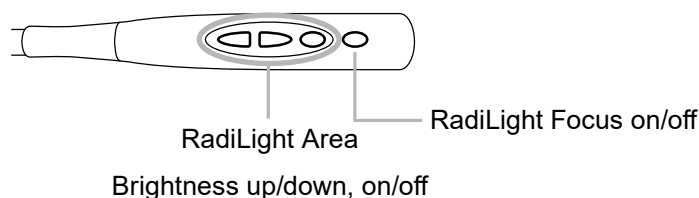

Attention


- The colors and brightness of LED lights are not consistent even among the products of the same model.
- RadiLight Area may not sufficiently light up the room in the following environment.
 - If there is a relatively long distance between the product and the wall or ceiling
 - If the material used for the wall or ceiling is irreflexive or a dark color

Specifications

Interface		Dedicated cable (supplied)
Voltage, current rating		DC 5.0 V $\pm$ 0.25 V (supplied from the USB port of the monitor), 500 mA
Power consumption (maximum)		2.5 W
Net Weight		Approx. 370 g (Approx. 280 g: Excluding mounting bracket)
Dimensions	Back lighting	184 mm (W) x 185.5 mm (H) x 15.7 mm (D)
	Lighting around hands	15 mm (W) x 95 mm (H) x 14 mm (D)
Operating Environmental Requirements	Temperature	0 - 35 °C
	Humidity	20 % - 80 % (R.H., no condensation)
	Air pressure	540 hPa - 1060 hPa
Transportation/ Storage Environmental Requirements	Temperature	-20 - 60 °C
	Humidity	10 % - 90 % (R.H., no condensation)
	Air pressure	200 hPa - 1060 hPa

Outside Dimensions

Unit: mm

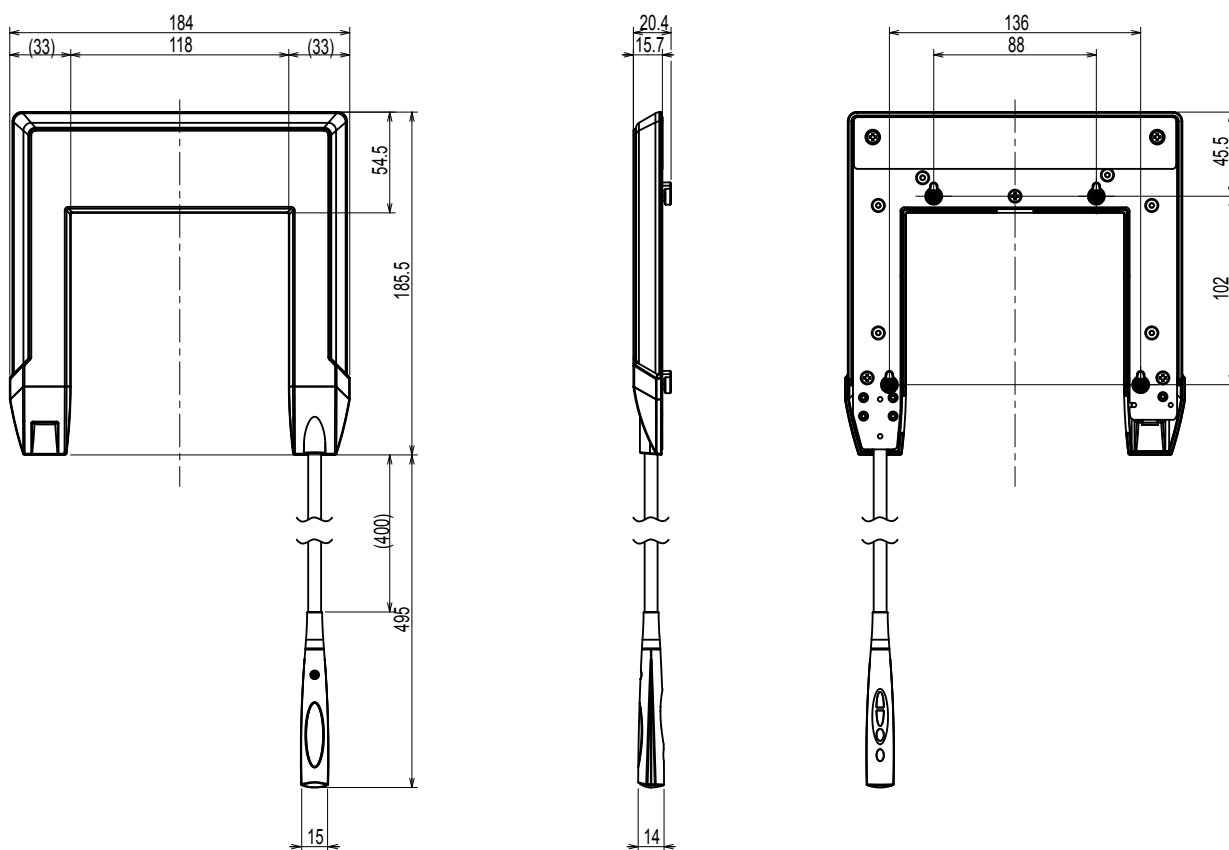Cleaning

In order to ensure the product remains clean and maintains a comfortable lighting intensity, periodic cleaning is recommended. To clean the illumination component, it is recommended that you use a soft cloth dampened with a small amount of water, and wipe gently. When using agents, always use the agents that are listed in the following table.

Chemicals that may be used for cleaning

Material name	Product name
Chlorhexidine	Hibitane
Sodium hypochlorite	Purelox
Benzalkonium chloride	Welpas
Alkyldiaminoethylglycine	Tego 51
Glutaral	Sterihyde
Glutaral	Cidex Plus28

Attention

- Do not use chemicals on a frequent basis. Chemicals such as alcohol and antiseptic solution may cause gloss variation, tarnishing, and fading of the cabinet or panel, and also quality deterioration of the image.
- Never use any thinner, benzene, wax, and abrasive cleaner, which may damage the cabinet or panel.
- Do not let chemicals come into direct contact with this product.
- A different cleaning agent must be used to clean this product and the monitor. When cleaning the monitor, always refer to the User's Manual of the monitor to check which agent should be used.

Limited Warranty

EIZO Corporation (hereinafter referred to as “**EIZO**”) and distributors authorized by EIZO (hereinafter referred to as the “**Distributors**”) warrant, subject to and in accordance with the terms of this limited warranty (hereinafter referred to as the “**Warranty**”), to the original purchaser (hereinafter referred to as the “**Original Purchaser**”) who purchased the product specified in this document (hereinafter referred to as the “**Product**”) from EIZO or Distributors, that EIZO and Distributors shall, at their sole discretion, either repair or replace the Product at no charge if the Original Purchaser becomes aware within the Warranty Period (defined below) that the Product malfunctions or is damaged in the course of normal use of the Product in accordance with the description in the instruction manual of the Product (hereinafter referred to as the “**User’s Manual**”).

The period of this Warranty is three (3) years from the date of purchase of the Product (hereinafter referred to as the “**Warranty Period**”). EIZO and Distributors shall bear no liability or obligation with regard to the Product in relation to the Original Purchaser or any third parties other than as provided under this Warranty.

EIZO and Distributors will cease to hold or store any parts (excluding design parts) of the Product upon expiration of five (5) years after the production of the Product is discontinued. In repairing the Product, EIZO and Distributors will use renewal parts which comply with our QC standards. If the unit cannot be repaired due to its condition or the stockout of a relevant part, EIZO and Distributors may offer the replacement by a product with equivalent performance instead of repairing it.

The Warranty is valid only in the countries or territories where the Distributors are located. The Warranty does not restrict any legal rights of the Original Purchaser.

Notwithstanding any other provision of this Warranty, EIZO and Distributors shall have no obligation under this Warranty whatsoever in any of the cases as set forth below:

- (a) When the limited warranty is not submitted;
- (b) Any defect of the Product on which the original serial number has been altered or removed;
- (c) Any defect of the Product caused by freight damage, modification, alteration, abuse, misuse, accident, incorrect installation, disaster, faulty maintenance and/or improper repair by third party other than EIZO and Distributors;
- (d) Malfunction of or damage to the product caused during transportation or movement, or due to the product being dropped after purchase;
- (e) Malfunction of or damage to the product caused by external factors such as fire, earthquakes, floods, lightning strikes or other acts of providence, and environment disruption or abnormal voltage;
- (f) Malfunction of or damage to the product caused by the product having been installed in an environment where it may be subjected to excessive shaking or impact such as in a vehicle or vessel;
- (g) Aging-related degradation of lighting (change of brightness or colors);
- (h) Damage to, and discoloration and degradation of exterior components;
- (i) Any normal deterioration of the product, particularly that of consumables, accessories, and/or attachments (e.g. cables, User’s Manual, etc.); and
- (j) Any incompatibility of the Product due to possible technical innovations and/or regulations.

To obtain service under the Warranty, the Original Purchaser must deliver the Product, freight prepaid, in its original package or other adequate package affording an equal degree of protection, assuming the risk of damage and/or loss in transit, to the local Distributor. The Original Purchaser must present proof of purchase of the Product and the date of such purchase when requesting services under the Warranty.

The Warranty Period for any replaced and/or repaired product under this Warranty shall expire at the end of the original Warranty Period.

EIZO OR DISTRIBUTORS ARE NOT RESPONSIBLE FOR ANY DAMAGE TO, OR LOSS OF, DATA OR OTHER INFORMATION STORED IN ANY MEDIA OR ANY PART OF ANY PRODUCT RETURNED TO EIZO OR DISTRIBUTORS FOR REPAIR.

EIZO AND DISTRIBUTORS MAKE NO FURTHER WARRANTIES, EXPRESSED OR IMPLIED, WITH RESPECT TO, INCLUDING, WITHOUT LIMITATION, THE PRODUCT AND ITS QUALITY, PERFORMANCE, MERCHANTABILITY OR FITNESS FOR ANY PARTICULAR USE. IN NO EVENT SHALL EIZO OR DISTRIBUTORS BE LIABLE FOR ANY INCIDENTAL, INDIRECT, SPECIAL, CONSEQUENTIAL OR OTHER DAMAGE WHATSOEVER (INCLUDING, WITHOUT LIMITATION, DAMAGES FOR LOSS OF PROFIT, BUSINESS INTERRUPTION, LOSS OF BUSINESS INFORMATION, OR ANY OTHER PECUNIARY LOSS) ARISING OUT OF THE USE OR INABILITY TO USE THE PRODUCT OR IN ANY CONNECTION WITH THE PRODUCT, WHETHER BASED ON CONTRACT, TORT, NEGLIGENCE, STRICT LIABILITY OR OTHERWISE, EVEN IF EIZO OR DISTRIBUTORS HAVE BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. THIS EXCLUSION ALSO INCLUDES ANY LIABILITY WHICH MAY ARISE OUT OF THIRD PARTY CLAIMS AGAINST THE ORIGINAL PURCHASER. THE ESSENCE OF THIS PROVISION IS TO LIMIT THE POTENTIAL LIABILITY OF EIZO AND DISTRIBUTORS ARISING OUT OF THIS LIMITED WARRANTY AND/OR SALES.

Appendix / Anhang / Annexe / Приложение

Information for Radio Interference

FCC Declaration of Conformity

We, the Responsible Party

EIZO Inc.

5710 Warland Drive, Cypress, CA 90630

Phone: (562) 431-5011

declare that the product

Trade name: EIZO

Model: RadiLight

is in conformity with Part 15 of the FCC Rules. Operation of this product is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures.

- * Reorient or relocate the receiving antenna.
- * Increase the separation between the equipment and receiver.
- * Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- * Consult the dealer or an experienced radio/TV technician for help.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note

Use the attached specified cable below with this product so as to keep interference within the limits of a Class B digital device.

- Shielded Signal Cable (enclosed)

Canadian Notice

This Class B digital apparatus complies with Canadian ICES-003.

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Recycling Information

This product, when disposed of, is supposed to be collected and recycled according to your country's legislation to reduce environmental burden. When you dispose of this product, please contact a distributor or an affiliate in your country.

The contact addressees are listed on the EIZO website below.

www.eizoglobal.com

For recycling information for customers in Switzerland, please refer to the following website.

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Copyright © 2021 - 2025 EIZO Corporation. All rights reserved.