



Installation Manual

RadiForce® MX194 Color LCD Monitor

Important

Please read this "Installation Manual", and "Instructions for Use" (separate volume) carefully to familiarize yourself with safe and effective usage.

-
- Refer to the "Instructions for Use" for information on the installation / connection of the monitor.
 - For the latest product information including the "Installation Manual", refer to our web site :
<http://www.eizoglobal.com>
-

This product has been adjusted specifically for use in the region to which it was originally shipped. If operated outside this region, the product may not perform as stated in the specifications.

No part of this manual may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, or otherwise, without the prior written permission of EIZO Corporation. EIZO Corporation is under no obligation to hold any submitted material or information confidential unless prior arrangements are made pursuant to EIZO Corporation's receipt of said information. Although every effort has been made to ensure that this manual provides up-to-date information, please note that EIZO monitor specifications are subject to change without notice.


CONTENTS


CONTENTS	3
Chapter 1 Basic Adjustment / Setting	4
1-1. How to Use Buttons	4
1-2. Switching the Input Signal.....	5
1-3. Switching Display Mode (CAL Switch Mode).....	5
1-4. Viewing the Monitor Information	6
Chapter 2 Advanced Adjustment / Setting	7
2-1. Basic Operation of the Setting Menu	7
2-2. Setting Menu Functions	9
● CAL Switch Mode.....	9
● Signal	12
● Preferences	13
● Languages	14
● Information	14
Chapter 3 Administrator Settings	15
3-1. Basic Operation of the "Administrator Settings" menu.....	15
3-2. "Administrator Settings" Menu Functions	16
● On-Screen Logo.....	16
● Key Lock.....	16
● DP Power Save	16
● Grayscale Warning.....	16
Chapter 4 Troubleshooting	17
4-1. No Picture	17
4-2. Imaging Problems (for Both Digital and Analog)	17
4-3. Imaging Problems (for Analog Only)	18
4-4. Other Problems	18
Chapter 5 Reference	19
5-1. Attaching the Optional Arm.....	19
5-2. Main Default Settings	20
Appendix	21
Trademark	21
License	21
LIMITED WARRANTY.....	22
Recycling Information	23






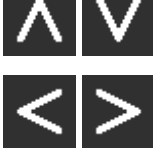


Chapter 1 Basic Adjustment / Setting

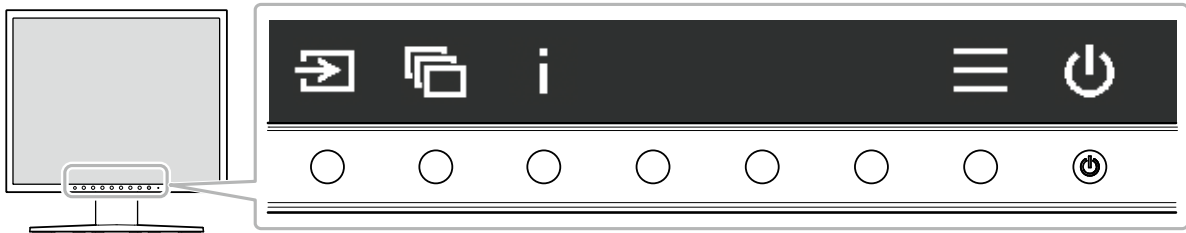1-1. How to Use Buttons


1. Displaying the operation guide

Press any button (excluding ).
The operation guide appears on the screen.




Display	Function
	Turns the power on or off.
	Displays the Setting menu. The Setting menu is used to configure advanced monitor settings.
	Displays the monitor information.
	Switches display modes.
	Switches input signals.
	Selects an item.
	Applies the selected content.
	Cancels the selection.

2. Adjusting / setting

1. Press a button for the icon to set.
The menu appears.
2. Operate each button to make the setting. If necessary, select  to apply the setting.




3. Exiting


Select  to exit the menu.

Note

- The operation guide will automatically disappear after a few seconds if no buttons are operated.

1-2. Switching the Input Signal

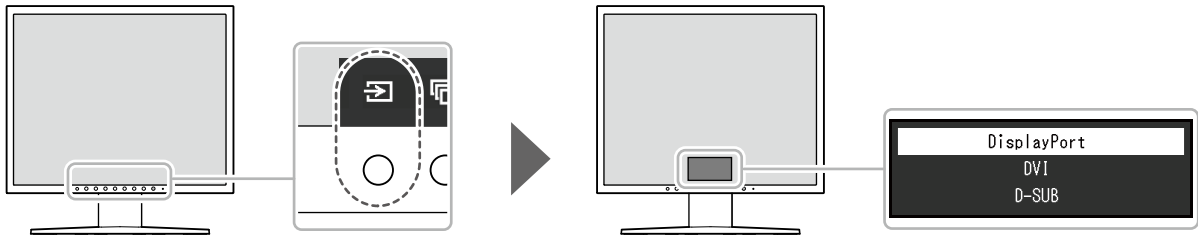


When a monitor has multiple signal inputs, the signal to display on-screen can be changed. Every time  is selected, the signal connector to display is changed in sequence and displayed.  or  also can be used to change modes while the menu appears.




Note

- When two or more computers are connected, select  to change the computer displayed on the screen.

1-3. Switching Display Mode (CAL Switch Mode)

This function allows easy selection of a display mode according to the content displayed on the monitor. The display mode changes in sequence every time  is selected.  or  also can be used to change modes while the menu appears.

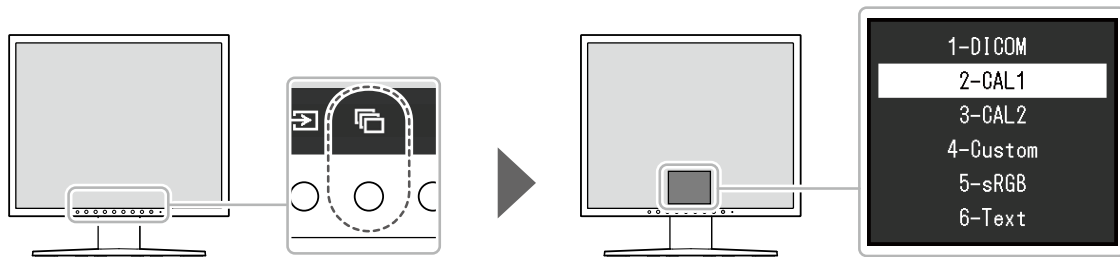


√: Can be calibrated

Mode	Purpose	
1-DICOM	Enables display using DICOM® Part 14-compliant grayscale characteristics.	√
2-CAL1	Displays the calibration status using calibration software.	√
3-CAL2		√
4-Custom	Select this to define a desired set of parameter settings.	-
5-sRGB	Suitable for color matching with sRGB compatible peripherals.	-
6-Text	Suitable to display documents, spreadsheets, or other text.	-

1-4. Viewing the Monitor Information

Monitor information (product name, serial number, usage time, and input signal) can be checked.

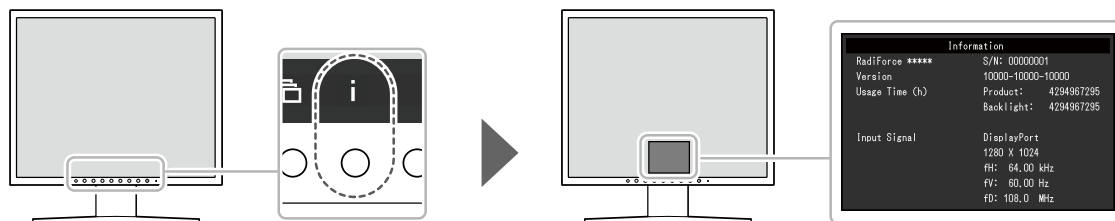


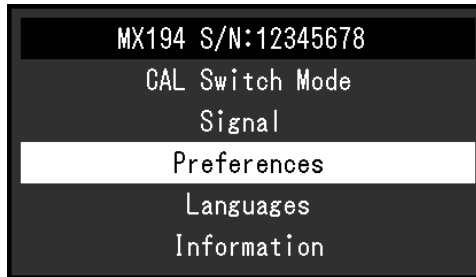
Chapter 2 Advanced Adjustment / Setting

This chapter describes the advanced monitor adjustment and setting procedures using the Setting menu.




2-1. Basic Operation of the Setting Menu


1. Displaying the Setting menu

1. Select  from the operation guide.
The Setting menu appears.






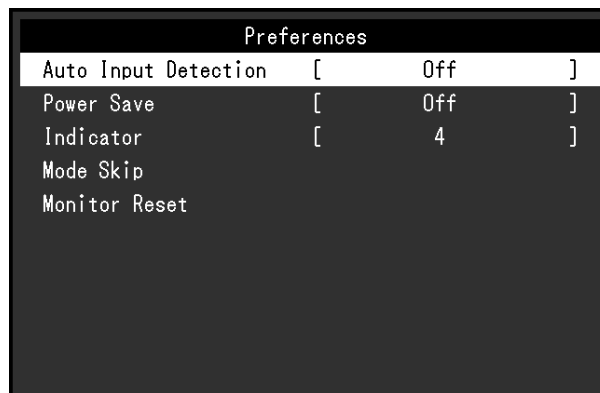
2. Adjusting / setting

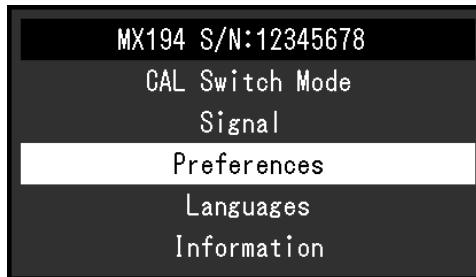1. Select a menu to adjust / set using  , then select .




The advanced menu appears.

2. Select an item to adjust / set using  , then select .



The Adjustment / Setting menu appears.

3. Adjust / set the selected item using **<** **>**, and then select **✓**.

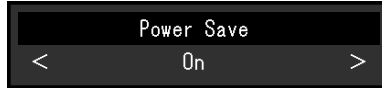

3. Exiting

Select **✕** several times to exit the Setting menu.

2-2. Setting Menu Functions

Note

- The display status of the monitor may change unexpectedly due to an operating error or unexpected setting change. Using the monitor with the control buttons locked is recommended after adjusting or setting the monitor. For details on how to set, refer to “Key Lock” (page 16).

● CAL Switch Mode

CAL Switch Mode (Custom)	
Mode	[Custom]
Brightness	[100]
Contrast	[50]
Temperature	[Off]
Gamma	
Hue	
Saturation	
Gain	
Reset	

Mode

This function allows easy selection of a CAL Switch mode (display mode) according to the content displayed on the monitor. For the features of each display mode, refer to “1-3. Switching Display Mode (CAL Switch Mode)” (page 5).

Setting Range

DICOM, CAL1, CAL2, Custom, sRGB, Text

Functions that can be adjusted differ depending on the display mode.

√: Adjustable -: Not adjustable

Function	CAL Switch Mode					
	DICOM	CAL1	CAL2	Custom	sRGB	Text
Brightness	√	√	√	√	√	√
Contrast	-	-	-	√	√	√
Temperature	-	-	-	√	√	√
Gamma	-	-	-	√	√	√
Hue	-	-	-	√	√	√
Saturation	-	-	-	√	√	√
Gain	-	-	-	√	√	√
Reset	√	√	√	√	√	√

Attention

- It takes about 30 minutes for the monitor display to stabilize. Please wait 30 or more minutes after the monitor power has been turned on, and then adjust the monitor.
- Perform the range adjustment first when adjusting color for analog signals (see “Auto Range Adjust” (page 12)).
- Each monitor has individual differences. For analog signals, if the same image is displayed side by side on multiple monitors, even the same image may appear different colors. Perform the range adjustment first when matching the colors of multiple monitors (see “Auto Range Adjust” (page 12)).

Brightness

The screen brightness is adjusted by changing the brightness of the backlight (light source from the LCD back panel).

Setting Range

0 % to 100 %

Note

- If the image is too dark even when the brightness is set to 100 %, increase the contrast value.
-

Contrast

The brightness of the screen is adjusted by varying the video signal level.

Setting Range

0 % to 100 %

Note

- When the contrast is set to 100 %, all the color gradations are shown.
 - When adjusting the monitor, it is recommended to perform the brightness adjustment which may not lose the grayscale characteristics, prior to the contrast adjustment.
 - Perform the contrast adjustment in the following cases.
 - If the screen is too bright even when the brightness is set to 0 %
-

Temperature

The color temperature can be adjusted.

The color temperature is normally used to express the color of "White" and/or "Black" by a numerical value. The value is expressed in degrees "K" (Kelvin).

The screen becomes reddish at a low color temperature, and bluish at a high color temperature, like a flame temperature. The gain preset values are set for each color temperature setting value.

Setting Range

Native, 6000 K to 15000 K (in increments of 100 K), sRGB

Note

- The value shown in "K" is available only as reference.
 - Selecting "Gain" allows you to perform more advanced adjustment (see ["Gain" \(page 11\)](#)).
 - If you set to "Off", the image is displayed in the preset color of the monitor (Gain: 100 for each RGB).
 - When gain is changed, the color temperature is changed to "Off".
-

Gamma

Adjust the gamma. The brightness of the monitor varies depending on the input signal, however, the variation rate is not simply proportional to the input signal. The control performed to keep the balance between the input signal and the brightness of the monitor is called "Gamma correction".

Setting Range

1.6 to 2.7, sRGB

Hue

Adjust the hue.

Setting Range

-100 to 100

Note

- Using this function may make some color gradations unavailable for display.
-

Saturation

Adjust the color saturation.

Setting Range

-100 to 100

Attention

- Using this function may make some color gradations unavailable for display.
-

Note

- The minimum value (-100) changes the screen to monochrome.
-

Gain

The brightness of each color component red, green, and blue is called Gain. You can change the hue of "white" by adjusting gain.

Setting Range

0 % to 100 %

Attention

- Using this function may make some color gradations unavailable for display.
-

Note

- The gain value changes according to the color temperature.
 - When gain is changed, the color temperature is changed to "User".
-

Reset

Resets any setting value for the currently selected display mode back to the default settings.

● Signal

Screen flickering, display position, and size can be adjusted when the analog signal is input (the D-Sub cable is connected).

Auto Screen Adjust

Screen flickering, display position, and size can be adjusted automatically. Selecting “Auto Screen Adjust” displays a message. Select “Yes” to enable this function.

Note

- The automatic screen adjustment function works correctly when an image is fully displayed over the entire displayable area of the screen. They do not work properly in the below cases.
 - When an image is displayed only on a part of the screen (command prompt window, for example)
 - When a black background (wallpaper, etc.) is in use
 - Also, these functions cannot work properly in some graphic boards.
 - When a signal is input into the monitor for the first time or when a resolution or vertical / horizontal scan frequency not used before is set, the screen is adjusted automatically (only for signals with a resolution of 800 × 600 or higher).
-


Auto Range Adjust

Every color gradation (0 to 255) can be displayed automatically by adjusting the adjustable signal output level.

Selecting “Auto Range Adjust” displays a message. Select “Yes” to enable this function.

Clock

Flickering of vertical lines on the screen or part of the screen can be reduced.

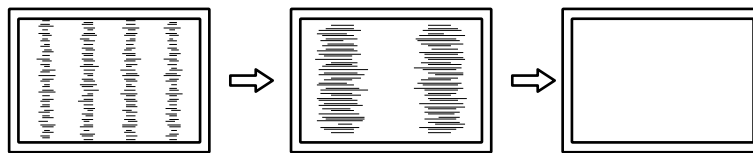


Note

- Use **▲** **▼** to make fine adjustments so as not to miss any adjustment points.
-

Phase

Flickering and blurring over the entire screen can be reduced.

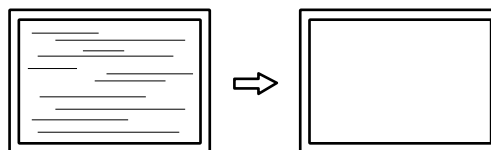


Note


- Flickering or blurring may not be eliminated completely depending on your PC or graphics board.
 - After setting, adjust the “Clock” again if vertical bars appear.
-

Hor. Position / Ver. Position

The display position (horizontal and vertical) of the image misaligned can be adjusted.

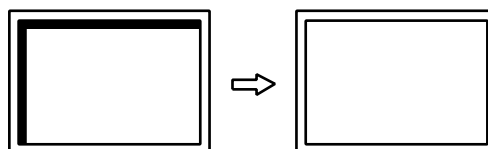

● Preferences

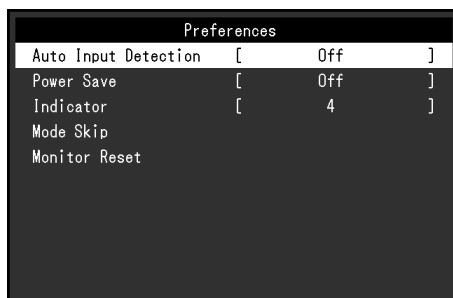


Auto Input Detection

This product automatically detects the connector through which a signal is input, and displays the signal on the screen.

Setting Range

On, Off

Setting	Function
On	Automatically detects the connector through which a signal is input, and displays the signal on the screen. If the PC that is outputting the signal displayed on the screen enters the power saving mode or is turned off when multiple PCs are connected, any other available signal is displayed automatically.
Off	The monitor displays the signal from the selected connector regardless of whether a signal is input or not. Select the input signal to display using  . For more information, see "1-2. Switching the Input Signal" (page 5).

Note

- When the power cable is reconnected, the signal of the connector to which the signal is impressed is displayed regardless of the settings.
- If "On" is selected, the monitor enters the power saving mode after all the PCs have entered the power saving mode or have been switched off.

Power Save

This function allows you to set the monitor into the power saving mode according to the PC status. When the monitor has shifted to the power saving mode, the images on the screen are not displayed.

Setting Range

On, Off

Setting	Function
On	The monitor shifts to the power saving mode synchronously with the state of the PC.
Off	The monitor does not shift to the power saving mode regardless of the status of the PC or signal input.

Note

- Disconnecting the power outlet cuts the power consumption completely, when not using the monitor.
- The monitor changes to power saving mode about 15 seconds after detecting no input signal.

Indicator

This function allows you to set the power indicator (green) ON / OFF in the monitor ON condition. The larger the set value, the brighter the power indicator lights up. On the other hand, the smaller the value, the darker it becomes. The "Off" setting turns off the power switch indicator.

Setting Range

1 to 7, Off

Mode Skip

When selecting the display mode, the specific mode becomes unavailable (skipped). Use this function to limit the display modes to appear, or avoid changing the display status randomly.

When "Mode skip" is selected, all display modes appear. Select the mode to be unavailable and select "Skip" for it.

Attention

- Not all modes can be disabled (skipped). Set at least one mode to "-" (available).
-

Monitor Reset

Restore all settings to their default values (except for Input Signal and "Administrator Settings").

● Languages

This function allows you to select a language for the Setting menu or messages.


Selectable languages

English / German / French / Spanish / Italian / Swedish / Japanese / Simplified Chinese / Traditional Chinese

● Information

Monitor information (product name, serial number, usage time, and input signal) can be checked.

Example:

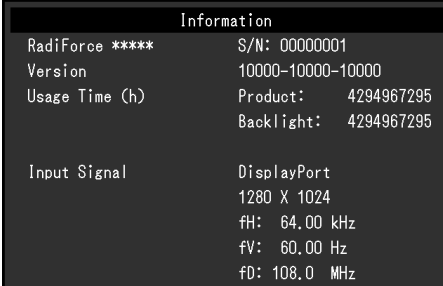

The screenshot shows a dark-themed menu titled "Information". It is divided into two sections. The first section lists: RadiForce ***** (S/N: 00000001), Version (10000-10000-10000), Usage Time (h) (Product: 4294967295, Backlight: 4294967295). The second section lists: Input Signal (DisplayPort, 1280 X 1024, fH: 64.00 kHz, fV: 60.00 Hz, fD: 108.0 MHz).

Information	
RadiForce *****	S/N: 00000001
Version	10000-10000-10000
Usage Time (h)	Product: 4294967295
	Backlight: 4294967295
Input Signal	DisplayPort
	1280 X 1024
	fH: 64.00 kHz
	fV: 60.00 Hz
	fD: 108.0 MHz


Chapter 3 Administrator Settings

This chapter describes how to configure monitor operation using the "Administrator Settings" menu. This menu is for administrators. Configuration on this menu is not required for normal monitor use.


3-1. Basic Operation of the "Administrator Settings" menu

1. Menu display



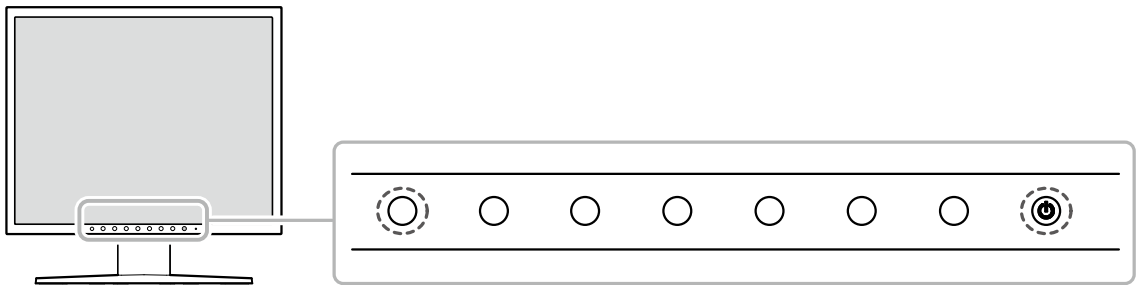

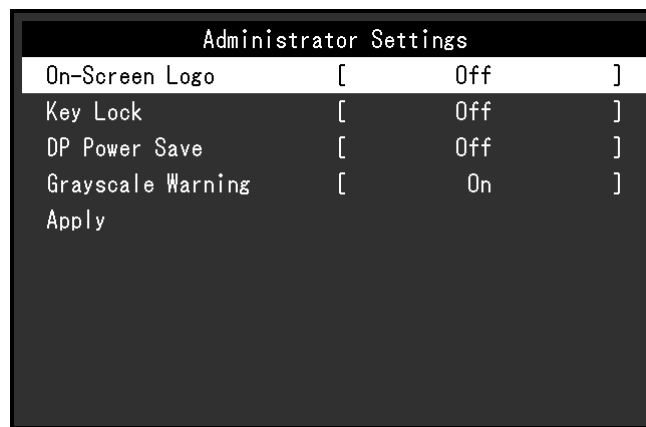

1. Press  to turn the monitor off.
2. While pressing the leftmost button, press  for more than 2 seconds to turn on the monitor.




"Administrator Settings" appears.



2. Setting

1. Select an item to set using  , and then select .
2. Set the item using   and select .

3. Applying and exiting

1. Select "Apply", and then select .

3-2. "Administrator Settings" Menu Functions

● On-Screen Logo

You can select whether to display or hide the EIZO logo when the monitor starts.

Setting Range


On, Off

● Key Lock

Adjusted or set states can be made unchangeable.

Setting Range

Off, Menu, All

Setting	Lockable functions
Off	None (All buttons are enabled.)
Menu	Adjusting or setting using the Setting menu
All	All buttons except 

● DP Power Save

Connecting a PC to the DisplayPort connector enables DisplayPort communication, and thus consumes high power even in the power saving mode. The communication can be disabled by setting "DP Power Save" to "On". This reduces power consumption in the power saving mode.

Setting Range

On, Off

Attention

- If "On" is selected, window and icon positions may shift when the monitor is powered back on or returned from the power saving mode. In such cases, set this function to "Off".
-

● Grayscale Warning

It can be specified whether to display the Grayscale Display Error message to indicate that a 6-bit (64-grayscale) signal has been detected when the DisplayPort signal is input at the recommended resolution. If the error message appears, disconnect the monitor's power cord and then connect it again. If it does not work, check the settings of the computer and graphics board.

Setting Range





On, Off

Attention

- Always use with this set to "On" (the default setting is "On").
 - Set this to "Off" only when using 6bit (64 grayscale) display.
-

Chapter 4 Troubleshooting




4-1. No Picture

Problem	Possible cause and remedy
1. No picture <ul style="list-style-type: none"> Power indicator does not light 	<ul style="list-style-type: none"> Check whether the power cord is connected properly. Press .
<ul style="list-style-type: none"> Power indicator lights green. 	<ul style="list-style-type: none"> Increase "Brightness", "Contrast", or "Gain" in the Setting menu (see "CAL Switch Mode" (page 9)).
<ul style="list-style-type: none"> Power indicator lights orange. 	<ul style="list-style-type: none"> Switch the input signal with . Move the mouse or press any key on the keyboard. Check whether the PC is turned on. This problem may occur when the PC is connected via the DisplayPort connector. Reconnect the power cord of the monitor, or reboot the PC.
2. The message below appears.	This message appears when the signal is not input correctly even when the monitor functions properly.
<ul style="list-style-type: none"> This message appears when no signal is input. Example: <div data-bbox="271 918 646 1052" style="border: 1px solid black; padding: 5px; text-align: center; margin: 10px 0;">  <p>DVI Signal Error</p> </div>	<ul style="list-style-type: none"> The message shown left may appear, because some PCs do not output the signal soon after power-on. Check whether the PC is turned on. Check whether the signal cable is connected properly. Switch the input signal with . Check whether the PC is configured to meet the resolution and vertical scan frequency requirements of the monitor (see "Compatible Resolutions" in the Instructions for Use). Reboot the PC. Change to the appropriate setting using the graphics board's utility. For details, refer to the User's Manual of the graphics board.

4-2. Imaging Problems (for Both Digital and Analog)

Problem	Possible cause and remedy
1. The screen is too bright or too dark.	<ul style="list-style-type: none"> Use "Brightness" or "Contrast" in the Setting menu to adjust it (see "CAL Switch Mode" (page 9)). (The LCD monitor backlight has a limited life span. When the screen becomes dark or begins to flicker, contact your local EIZO representative.)
2. Characters are blurred.	<ul style="list-style-type: none"> Check whether the PC is configured to meet the resolution and vertical scan frequency requirements of the monitor (see "Compatible Resolutions" in the Instructions for Use).
3. Afterimages appear	<ul style="list-style-type: none"> Afterimages are particular to LCD monitors. Avoid displaying the same image for a long time. Use the screen saver or power saving function to avoid displaying the same image for extended periods of time.
4. Green / red / blue / white dots or defective dots remain on the screen.	<ul style="list-style-type: none"> This is due to LCD panel characteristics and not a malfunction.
5. Interference patterns or traces created when selecting panels remain on the screen.	<ul style="list-style-type: none"> Display a white or black image over the entire screen. The symptom may disappear.

4-3. Imaging Problems (for Analog Only)

Problem	Possible cause and remedy
<p>1. Display position is incorrect.</p> 	<ul style="list-style-type: none"> Use "Hor. Position" and "Ver. Position" in the Setting menu to correct the image position (see "Hor. Position / Ver. Position" (page 12)). If the problem persists, use the graphics board's utility if available to change the display position.
<p>2. Vertical bars appear on the screen or a part of the image is flickering.</p> 	<ul style="list-style-type: none"> Use "Clock" in the Setting menu to adjust it (see "Clock" (page 12)).
<p>3. Whole screen is flickering or blurring.</p> 	<ul style="list-style-type: none"> Use "Phase" in the Setting menu to adjust it (see "Phase" (page 12)).

4-4. Other Problems

Problem	Possible cause and remedy
<p>1. The Setting menu does not appear.</p>	<ul style="list-style-type: none"> Check whether the operation lock function is active (see "Key Lock" (page 16)).
<p>2. The auto adjustment function does not work.</p>	<ul style="list-style-type: none"> This function does not work during digital signal input. This function works correctly when an image is displayed over the entire displayable area of the Windows screen, etc. It does not work properly when an image is displayed only on a part of the screen (command prompt window, for example) or when a black background (wallpaper, etc.) is in use. This function does not work correctly with some graphics boards.
<p>3. The monitor connected with the USB cable is not detected.</p>	<ul style="list-style-type: none"> Check whether the USB cable is connected correctly. Reboot the PC. Check whether the PC and OS are USB compliant. (For USB compliance of the respective devices, consult their manufacturers.) Check the PC's BIOS setting for USB when using Windows. (Refer to the User's Manual of the PC for details.)

Chapter 5 Reference

5-1. Attaching the Optional Arm

An optional arm (or an optional stand) can be attached by removing the stand section.

Attention

- When attaching an arm or stand, follow the instructions of the respective User's Manual.
- When using another manufacturer's arm or stand, confirm the following in advance and select one conforming to the VESA standard.
 - Clearance between the screw holes: 100 mm × 100 mm
 - Strong enough to support weight of the monitor unit (excluding the stand) and attachments such as cables
- When using another manufacturer's arm or stand, use the following screws to secure it.
 - Screws fixing the stand to the monitor
- When using an arm or stand, attach it to meet the following tilt angles of the monitor.
 - 45° up to 45° down
- Connect the cables after attaching an arm or stand.
- Do not move the removed stand up and down. Doing so may result in injury or equipment damage.
- If you drop the monitor, arm, or stand, injury or equipment damage may result.
- Periodically check the tightness of the screws. If not sufficiently tight, the monitor may detach from the arm, which may result in injury or equipment damage.
- When installing the monitor in portrait mode, turn the monitor screen 90° in clockwise direction and place it.

Attachment procedure


1. To prevent damaging the panel surface, lay the monitor on a soft cloth spread on a stable surface with the panel surface facing down.

2. Remove the stand.

Using a screwdriver, loosen the screws securing the unit and the stand.

3. Attach the arm or stand to the monitor.

Use the screws that you removed in step 2 to attach the arm (or stand) to the monitor.

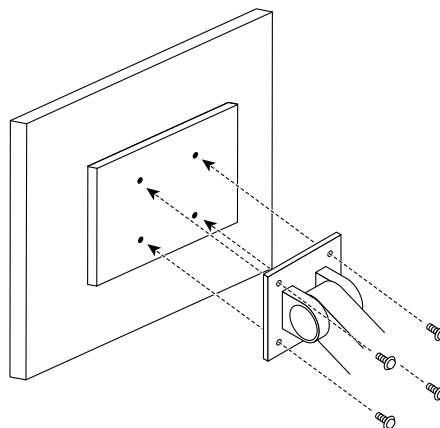

5-2. Main Default Settings

CAL Switch Mode: Factory default display setting is "1-DICOM".

Mode	Brightness	Temperature	Gamma value
1-DICOM	180 cd / m ²	7500 K	DICOM GSDF
2-CAL1	180 cd / m ²	7500 K	2.2
3-CAL2	120 cd / m ²	7500 K	DICOM GSDF
4-Custom	Approx. 300 cd / m ²	7500 K	2.2
5-sRGB	Approx. 250 cd / m ²	sRGB	sRGB
6-Text	Approx. 100 cd / m ²	6500 K	2.2

Others

Auto Input Detection	Off
Power Save	On
Indicator	4
Mode Skip	- (Display all)
Language	English
On-Screen Logo	Off* ¹
Key Lock	Off* ¹
DP Power Save	Off* ¹
Grayscale Warning	On* ¹

*¹ These menus cannot be initialized by executing "Monitor Reset" (see ["Monitor Reset"](#) (page 14)).

Appendix

Trademark

The terms HDMI and HDMI High-Definition Multimedia Interface, and the HDMI Logo are trademarks or registered trademarks of HDMI Licensing, LLC in the United States and other countries.

The DisplayPort Compliance Logo and VESA are registered trademarks of the Video Electronics Standards Association.

The SuperSpeed USB Trident Logo is a registered trademark of USB Implementers Forum, Inc.



DICOM is the registered trademark of the National Electrical Manufacturers Association for its standards publications relating to digital communications of medical information.

Kensington and Microsaver are registered trademarks of ACCO Brands Corporation.

Thunderbolt is a trademark of Intel Corporation in the United States and/or other countries.

Microsoft and Windows are registered trademarks of Microsoft Corporation in the United States and other countries.

Adobe is a registered trademark of Adobe Systems Incorporated in the United States and other countries.

Apple, macOS, Mac OS, OS X, Macintosh and ColorSync are registered trademarks of Apple Inc.

EIZO, the EIZO Logo, ColorEdge, CuratOR, DuraVision, FlexScan, FORIS, RadiCS, RadiForce, RadiNET, Raptor and ScreenManager are registered trademarks of EIZO Corporation in Japan and other countries.

ColorEdge Tablet Controller, ColorNavigator, EcoView NET, EIZO EasyPIX, EIZO Monitor Configurator, EIZO ScreenSlicer, G-Ignition, i•Sound, Quick Color Match, RadiLight, Re/Vue, Screen Administrator, Screen InStyle and UniColor Pro are trademarks of EIZO Corporation.

All other company and product names are trademarks or registered trademarks of their respective owners.

License

The bitmap font used for this product is designed by Ricoh Industrial Solutions Inc.

LIMITED WARRANTY

EIZO Corporation (hereinafter referred to as “**EIZO**”) and distributors authorized by EIZO (hereinafter referred to as the “**Distributors**”) warrant, subject to and in accordance with the terms of this limited warranty (hereinafter referred to as the “**Warranty**”), to the original purchaser (hereinafter referred to as the “**Original Purchaser**”) who purchased the product specified in this document (hereinafter referred to as the “**Product**”) from EIZO or Distributors, that EIZO and Distributors shall, at their sole discretion, either repair or replace the Product at no charge if the Original Purchaser becomes aware within the Warranty Period (defined below) that (i) the Product malfunctions or is damaged in the course of normal use of the Product in accordance with the description in the instruction manual of the Product (hereinafter referred to as the “**User’s Manual**”), or (ii) the LCD panel and brightness of the Product cannot maintain the recommended brightness specified in the User’s Manual in the course of normal use of the Product in accordance with the description of the User’s Manual.

The period of this Warranty is five (5) years from the date of purchase of the Product (hereinafter referred to as the “**Warranty Period**”). However, the brightness of the Product shall be warranted only if the Product has been used within the recommended brightness described in the User’s Manual. The Warranty Period for the brightness is also limited to five (5) years from the date of purchase of the Product subject to the usage time being less than or equal to 10,000 hours (the brightness is 180 cd/m² and color temperature is 7500K). EIZO and Distributors shall bear no liability or obligation with regard to the Product in relation to the Original Purchaser or any third parties other than as provided under this Warranty.

EIZO and Distributors will cease to hold or store any parts (excluding design parts) of the Product upon expiration of seven (7) years after the production of the Product is discontinued. In repairing the monitor, EIZO and Distributors will use renewal parts which comply with our QC standards. If the unit cannot be repaired due to its condition or the stockout of a relevant part, EIZO and Distributors may offer the replacement by a product with equivalent performance instead of repairing it.

The Warranty is valid only in the countries or territories where the Distributors are located. The Warranty does not restrict any legal rights of the Original Purchaser.

Notwithstanding any other provision of this Warranty, EIZO and Distributors shall have no obligation under this Warranty whatsoever in any of the cases as set forth below:

- (a) Any defect of the Product caused by freight damage, modification, alteration, abuse, misuse, accident, incorrect installation, disaster, faulty maintenance and/or improper repair by third party other than EIZO and Distributors;
- (b) Any incompatibility of the Product due to possible technical innovations and/or regulations;
- (c) Any deterioration of the sensor, including measurement value of the sensor;
- (d) Any defect of the Product caused by external equipments;
- (e) Any defect of the Product caused by the use under environmental conditions other than EIZO and Distributors’ estimation;
- (f) Any deterioration of the attachments of the Product (e.g. cables, User’s Manual, CD-ROM, etc.);
- (g) Any deterioration of the consumables, and/or accessories of the Product (e.g. batteries, remote controller, touch pen, etc.);
- (h) Any deformation, discoloration, and/or warp of the exterior of the product including that of the surface of the LCD panel;
- (i) Any defect of the Product caused by placement in a location where it may be affected by strong vibration or shocks;
- (j) Any defect of the Product caused by liquid leaking from battery;
- (k) Any deterioration of the Product caused by the use at higher brightness than the recommended brightness described in the User’s Manual;
- (l) Any deterioration of display performance caused by the deterioration of expendable parts such as the LCD panel and/or backlight, etc. (e.g. changes in brightness uniformity, changes in color, changes in color uniformity, defects in pixels including burnt pixels, etc.).

To obtain service under the Warranty, the Original Purchaser must deliver the Product, freight prepaid, in its original package or other adequate package affording an equal degree of protection, assuming the risk of damage and/or loss in transit, to the local Distributor. The Original Purchaser must present proof of purchase of the Product and the date of such purchase when requesting services under the Warranty.

The Warranty Period for any replaced and/or repaired product under this Warranty shall expire at the end of the original Warranty Period.

EIZO OR DISTRIBUTORS ARE NOT RESPONSIBLE FOR ANY DAMAGE TO, OR LOSS OF, DATA OR OTHER INFORMATION STORED IN ANY MEDIA OR ANY PART OF ANY PRODUCT RETURNED TO EIZO OR DISTRIBUTORS FOR REPAIR.

EIZO AND DISTRIBUTORS MAKE NO FURTHER WARRANTIES, EXPRESSED OR IMPLIED, WITH RESPECT TO THE PRODUCT AND ITS QUALITY, PERFORMANCE, MERCHANTABILITY OR FITNESS FOR ANY PARTICULAR USE. IN NO EVENT SHALL EIZO OR DISTRIBUTORS BE LIABLE FOR ANY INCIDENTAL, INDIRECT, SPECIAL, CONSEQUENTIAL OR OTHER DAMAGE WHATSOEVER (INCLUDING, WITHOUT LIMITATION, DAMAGES FOR LOSS OF PROFIT, BUSINESS INTERRUPTION, LOSS OF BUSINESS INFORMATION, OR ANY OTHER PECUNIARY LOSS) ARISING OUT OF THE USE OR INABILITY TO USE THE PRODUCT OR IN ANY CONNECTION WITH THE PRODUCT, WHETHER BASED ON CONTRACT, TORT, NEGLIGENCE, STRICT LIABILITY OR OTHERWISE, EVEN IF EIZO OR DISTRIBUTORS HAVE BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. THIS EXCLUSION ALSO INCLUDES ANY LIABILITY WHICH MAY ARISE OUT OF THIRD PARTY CLAIMS AGAINST THE ORIGINAL PURCHASER. THE ESSENCE OF THIS PROVISION IS TO LIMIT THE POTENTIAL LIABILITY OF EIZO AND DISTRIBUTORS ARISING OUT OF THIS LIMITED WARRANTY AND/OR SALES.

Recycling Information

Recycling Information

This product, when disposed of, is supposed to be collected and recycled according to your country's legislation to reduce environmental burden. When you dispose of this product, please contact a distributor or an affiliate in your country.

The contact addressees are listed on the EIZO website below.

<http://www.eizoglobal.com>

For recycling information for customers in Switzerland, please refer to the following website.

<http://www.swico.ch>

Informationen zum Thema Recycling

Dieses Produkt muss gemäß den Vorschriften Ihres Landes zur Entlastung der Umwelt recyclet werden. Wenden Sie sich bei der Entsorgung dieses Produkts an einen Verteiler oder eine Tochtergesellschaft in Ihrem Land.

Die Adressen zur Kontaktaufnahme sind auf der unten angegebenen Website von EIZO aufgeführt.

<http://www.eizoglobal.com>

Kunden in der Schweiz entnehmen Informationen zum Recycling der folgenden Website:

<http://www.swico.ch>

Informations sur le recyclage

Ce produit doit être jeté aux points de collecte prévus à cet effet et recyclé conformément à la législation de votre pays, afin de réduire l'impact sur l'environnement. Lorsque vous jetez ce produit, veuillez contacter un distributeur ou une société affiliée de votre pays.

Les adresses des distributeurs sont répertoriées sur le site Web EIZO ci-dessous.

<http://www.eizoglobal.com>

Pour les clients en Suisse, veuillez consulter le site Web suivant afin d'obtenir des informations sur le recyclage.

<http://www.swico.ch>

Información sobre reciclaje

Este producto debe desecharse y reciclarse según la legislación del país para reducir el impacto medioambiental. Cuando desee deshacerse de este producto, póngase en contacto con un distribuidor o una filial de su país.

Encontrará las direcciones de contacto en el sitio web de EIZO que se indica a continuación.

<http://www.eizoglobal.com>

Informazioni sul riciclaggio

Per lo smaltimento e il riciclaggio del presente prodotto, attenersi alle normative vigenti nel proprio paese per ridurre l'impatto ambientale. Per lo smaltimento, rivolgersi ad un distributore o un affiliato presenti nel proprio paese.

Gli indirizzi sono elencati nel sito Web EIZO riportato di seguito.

<http://www.eizoglobal.com>

Per informazioni sul riciclaggio per i clienti in Svizzera, consultare il sito Web riportato di seguito.

<http://www.swico.ch>

Återvinningsinformation

När denna produkt kasseras ska den hanteras och återvinnas enligt landets föreskrifter för att reducera miljöpåverkan. När du kasserar produkten ska du kontakta en distributör eller representant i ditt land.

Kontaktadresserna listas på EIZO-webbplatsen nedan.

<http://www.eizoglobal.com>

Πληροφορίες ανακύκλωσης

Το προϊόν αυτό, όταν απορρίπτεται, πρέπει να συλλέγεται και να ανακυκλώνεται σύμφωνα με τη νομοθεσία της χώρας σας έτσι ώστε να μην επιβαρύνει το περιβάλλον. Για να απορρίψετε το προϊόν, επικοινωνήστε με έναν αντιπρόσωπο ή μια θυγατρική εταιρεία στη χώρα σας.

Οι διευθύνσεις επικοινωνίας αναγράφονται στην τοποθεσία web της EIZO παρακάτω.

<http://www.eizoglobal.com>

Сведения по утилизации

По истечении срока службы данного продукта его следует принести на сборный пункт и утилизировать в соответствии с действующими предписаниями в вашей стране, чтобы уменьшить вредное воздействие на окружающую среду. Прежде чем выбросить данный продукт, обратитесь к дистрибьютору или в местное представительство компании в вашей стране.

Контактные адреса можно найти на веб-узле EIZO.

<http://www.eizoglobal.com>

Informatie over recycling

Wanneer u dit product wilt weggooien, moet het uit milieu-overwegingen worden verzameld en gerecycled volgens de betreffende wetgeving van uw land. Wanneer u dit product wilt weggooien, moet u contact opnemen met een distributeur of een partner in uw land.

De contactadressen worden vermeld op de volgende EIZO-website.

<http://www.eizoglobal.com>

Informação sobre reciclagem

Este produto, quando o deitar fora, deve ser recolhido e reciclado de acordo com a legislação do seu país para reduzir a poluição. Quando deitar fora este produto, contacte um distribuidor ou uma filial no seu país.

Os endereços de contacto estão listados no website do EIZO, abaixo.

<http://www.eizoglobal.com>

Oplysninger om genbrug

Denne produkt forventes ved bortskaffelse at blive indsamlet og genbrugt i overensstemmelse med lovgivningen i dit land for at reducere belastningen af miljøet. Når du bortskaffer denne produkt, skal du kontakte en distributør eller et tilknyttet selskab i dit land.

Adresserne på kontaktpersonerne er angivet på EIZO's websted nedenfor.

<http://www.eizoglobal.com>

Kierrätystä koskevia tietoja

Tuote tulee hävittää kierrättämällä maan lainsäädännön mukaisesti ympäristön kuormittumisen vähentämiseksi. Kun hävität tuotteen, ota yhteyttä jälleenmyyjään tai tytäryhtiöön maassasi.

Yhteystiedot löytyvät EIZOn Internet-sivustolta.

<http://www.eizoglobal.com>

Wykorzystanie surowców wtórnych

Ten produkt po zużyciu powinien być zbierany i przetwarzany zgodnie z krajowymi przepisami dotyczącymi ochrony środowiska. Wyrzucając ten produkt, należy skontaktować się z lokalnym dystrybutorem lub partnerem. Adresy kontaktowe można znaleźć we wskazanej poniżej witrynie internetowej firmy EIZO.

<http://www.eizoglobal.com>

Informace o recyklaci

Při likvidaci produktu musí být produkt vyzvednut a recyklován podle zákonů příslušné země, aby nedocházelo k zatěžování životního prostředí. Zbavujete-li se produktu, kontaktujte distributora nebo pobočku ve své zemi. Kontaktní adresy jsou uvedeny na následující webové stránce společnosti EIZO.

<http://www.eizoglobal.com>

Ringlussevõtu alane teave

Keskkonnakoormuse vähendamiseks tuleks kasutatud tooted kokku koguda ja võtta ringlusse vastavalt teie riigi seadustele. Kui kõrvaldate käesoleva toote, võtke palun ühendust turustaja või filiaaliga oma riigis.

Kontaktisikud on toodud EIZO veebilehel järgmisel aadressil:

<http://www.eizoglobal.com>

Újrahasznosítási információ

Jelen terméket, amikor eldobjuk, az illető ország törvényei alapján kell összegyűjteni és újrahasznosítani, hogy csökkentjük a környezet terhelését. Amikor ezt a terméket eldobja, kérjük lépjen kapcsolatba egy országon belüli forgalmazóval vagy társvállalkozással.

A kapcsolat címei az EIZO alábbi weboldalán található felsorolva:

<http://www.eizoglobal.com>

Podatki o recikliranju

Ob koncu uporabe odslužen izdelek izročite na zbirno mesto za recikliranje v skladu z okoljevarstveno zakonodajo vaše države. Prosimo, da se pri odstranjevanju izdelka obrnete na lokalnega distributerja ali podružnico.

Kontaktni naslovi so objavljeni na spodaj navedenih spletnih straneh EIZO:

<http://www.eizoglobal.com>

Informácie o recyklácii

Pri likvidácii musí byť tento výrobok recyklovaný v súlade s legislatívou vašej krajiny, aby sa znížilo zaťaženie životného prostredia. Keď sa chcete zbaviť tohto výrobku, kontaktujte prosím distribútora alebo pobočku vo vašej krajine.

Zoznam kontaktných adries je uvedený na nasledovnej webovej stránke firmy EIZO:

<http://www.eizoglobal.com>

Pārstrādes informācija

Utilizējot šo produktu, tā savākšana un pārstrāde veicama atbilstoši jūsu valsts likumdošanas prasībām attiecībā uz slodzes uz vidi samazināšanu. Lai veiktu šī produkta utilizāciju, sazinieties ar izplatītāju vai pārstāvniecību jūsu valstī.

Kontaktadreses ir norādītas zemāk minētajā EIZO tīmekļa vietnē.

<http://www.eizoglobal.com>

Informacija apie gražinamąjį perdirbimą

Šalinant (išmetant) šiuos gaminius juos reikia surinkti ir perdirbti gražinamuoju būdu pagal jūsų šalies teisės aktus, siekiant sumažinti aplinkos teršimą. Kai reikia išmesti šį gaminį, kreipkitės į jūsų šalyje veikiančią gaminių pardavimo atstovybę arba jos filialą.

Kontaktiniai adresatai nurodyti EIZO interneto svetainėje.

<http://www.eizoglobal.com>

Информация относно рециклиране

При изхвърлянето на този продукт се предлага събирането и рециклирането му съобразно законите на вашата страна за да се намали замърсяването на околната среда. Когато искате да се освободите от този продукт, моля свържете се с търговския му представител или със съответните органи отговарящи за това във вашата страна.

Данните за връзка с нас са описани на следния Интернет сайт на EIZO:

<http://www.eizoglobal.com>

Informație referitoare la reciclare

Acest produs, când debarasat, trebuie colectat și reciclat conform legislației țării respective ca să reducem sarcina mediului. Când se debarasează de acest produs, vă rugăm să contactați un distribuitor sau un afiliat al țării respective.

Adresele de contact le găsiți pe pagina de web al EIZO:

<http://www.eizoglobal.com>

معلومات عن اعادة التدوير

عند التخلص من هذا المنتج، من الضروري تجميعه واعادة تدويره وفقاً للقوانين المتبعة في بلدك لتقليل العبء المسلط على البيئة. عندما تتخلص من هذا المنتج، يرجى الاتصال بموزع أو طرف منتسب إليه في بلدك. عناوين الاتصال مدرجة في موقع ايزو EIZO ادناه على شبكة الانترنت.

<http://www.eizoglobal.com>

Gerı Dönüşüm Bilgisi

Bu ürünün, atılacağı zaman, ülkenizin çevre kirliliğinin azaltılması konusundaki mevzuatına göre toplanması ve yeniden değerlendirilmesi gerekmektedir. Ürünü atacağınız zaman lütfen ülkenizdeki bir distribütör veya ilgili kuruluşla temasa geçiniz.

İrtibat adresleri aşağıdaki EIZO web sitesinde verilmiştir.

<http://www.eizoglobal.com>

